Fix

Neuro Head Holder—8002046

INTENDED USE

The Neuro Head Holder is designed to position the head. This device is primarily intended for neuroradiology procedures (e.g. interventional procedures) in order to minimize patient's head motion. The Neuro Head Holder is intended to be used in a clinical environment by trained technicians and clinicians.

! NOTE ! United States Federal law restricts this device to sale by or on the order of a physician.

Patient Target Groups

Patients undergoing general medical or surgical procedures involving diagnostic imaging.

Intended Users

The intended user for the products is a person qualified in accordance with the requirements of the regulatory region.

GENERAL PRECAUTIONS

Environmental Conditions

- Operating temperature: max of 37°C (100°F)
- Operating Relative Humidity: 5% to 95% non-condensing
- Storage temperature range: -25°C to 70°C (-13°F to 158°F)
- Storage relative humidity: 5% to 95%

WARNING STATEMENTS

! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT PHILIPS HEALTHCARE HELPDESK.

! CAUTION ! EDGES OF HEAD PAD MAY ARTIFACT AT SOME IMAGING ANGLES.

SERIOUS INCIDENTS

Please report any serious incidents (e.g. incidents which result in or have the potential to result in death or serious injury) to both Qfix and your country's Competent Authority.

WARNING LABELS & DESCRIPTIONS

Refer to Qfix.com for a listing of symbols and their definitions.

LOAD RATING

! WARNING ! DO NOT EXCEED 136 KG (300 LB) PATIENT LOAD.

ALUMINUM EQUIVALENCE

Neuro Head Holder Pad: ~.45 mm of Al

Neuro Head Holder Shell: ~.35 mm of Al

FEATURES

Description

The product is designed to complement and enhance image quality, performance and productivity of cardiovascular x-ray equipment. The product supports the patient's head reducing the misalignment with the subtracted image and enhancing 3-D images. The product's simple design makes for easy installation, adjustment and removal and allows the user to focus on the needs of the patient.

MAINTENANCE

The device can be cleaned with a mild, non-abrasive cleaning or disinfecting solution. Do not spray or pour liquids onto the device. To clean, apply solution to clean cloth and wipe the surface. The following cleaning materials have been tested and found to be appropriate for cleaning the device surfaces. To disinfect the surface, refer to specific instructions from the cleaning agent manufacturer:

- Water
- 10% Clorox® Bleach Solution
- Isopropyl Alcohol
- Cidex® 2.4% Activated Dialdehyde Solution
- Soap and Water

DO NOT spray directly onto the Neuro Head Holder.

DO NOT leave the Neuro Head Holder in water overnight.

DO NOT use chlorine to clean the Neuro Head Holder.

Periodically check all fasteners for tightness.

OPERATING INSTRUCTIONS

Installation

Neuro Head Holder Installation

1. Hold the Neuro Head Holder with both hands, one holding the top and one holding the clamp. Slide the clamp onto the table at the front end of the table (Fig. 1).
2. Rotate the knob clockwise until the Neuro Head Holder is firmly fixed to the table (Fig. 2).

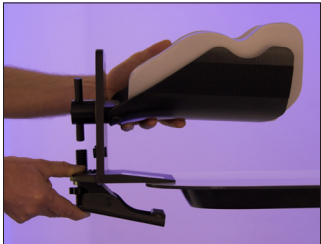
Fig. 1

Fig. 2

Set-Up

Neuro Head Holder Adjustment

1. Turn the knob counterclockwise to loosen the head shell, while holding the head shell with one hand and the knob with the other adjust the height accordingly by moving it up or down (Fig. 3)
2. Tighten knob. Loosen wing nut and adjust placement of drop stop to contact knob. Tighten wing nut to secure placement (Fig. 4).
3. To move the head shell laterally, loosen the knob and rotate the head shell to the right or left. (Fig 5).

! NOTE ! Adjustments may have to be made once the patient is in place.

4. Turn the knob clockwise to tighten into place. (Fig. 5).

Fig. 3

Fig. 4

Fig. 5

5. Position the patient on the table after you have installed the Neuro Head Holder. Readjust the height and orientation of the head shell as necessary. Use straps to fixate the patient securely. See correct positioning shown in Fig. 6.

! NOTE ! When positioning the patient, pay attention not to put the patient too high in the Neuro Head Holder (Fig 7). This will be uncomfortable and may cause artifact.

Fig. 6: CORRECT

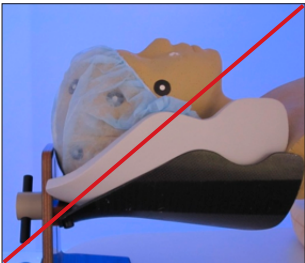
Fig. 7: NOT CORRECT

Removal

1. Remove the patient from the Neuro Head Holder.
2. Rotate the knob counterclockwise to loosen from the table (Fig. 8).
3. Holding the Neuro Head Holder with two hands, one holding the top and one holding the clamp, push down on the clamp to remove it from the table (Fig. 9).

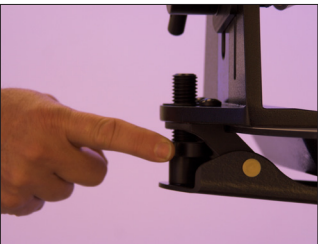
Fig. 8

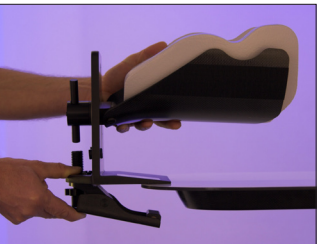
Fig. 9

PARTS LIST

Components	Qfix Part Number	Philips 12nc
Neuro Head Holder	8002046	4598 005 91071
Head pad & straps	8002206	4598 005 91091
Straps	8002205	4598 005 91101

CE EC REP Advena Limited Tower Business Centre, 2nd Flr Tower Street, Swatar, BKR 4013 Malta	UK CA UK Responsible Person Qfix Medical Limited 3rd Floor, Chancery House St. Nicholas Way Sutton, Surrey SM1 1JB United Kingdom
Made in the USA by Qfix 440 Church Rd, Avondale, Pennsylvania, USA +1 484-720-6053 www.Qfix.com	CH REP Raditec Medical AG Schlossberg 5a 5454 Bellikon Switzerland

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