PRODUCT GUIDE & USER MANUAL

RT-4550SS4 QUANTUM™ CT Overlay —
Siemens CT Scanner

 EC REP Advena Limited Tower Business Centre, 2nd Flr Tower Street, Swatar, BKR 4013 Malta	
 Made in the USA by Qfix 440 Church Rd, Avondale, Pennsylvania, USA +1 712-737-8688 www.CQmedical.com	CH REP Raditec Medical AG Schlossberg 5a 5454 Bellikon Switzerland

2004648_ML_1 / 2025-06

EN	English	PT-EU	Portuguese (Português)
DE	German (Deutsch)	RO	Romanian (Română)
EL	Greek (Ελληνικά)	RU	Russian (Русский язык)
ES	Spanish (Español)	SV	Swedish (Svenska)
FR	French (Français)	UK	Ukrainian (Українська)
IT	Italian (Italiano)	VI	Vietnamese (Tiếng Việt)
JA	Japanese (日本語)	ZH	Chinese (简体中文)
NL	Dutch (Nederlands)		
PT-BR	Portuguese BR (Português Brasil)		

EN: English

GENERAL PRECAUTIONS

WARNING STATEMENTS


! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT CQ MEDICAL AT +1 712-737-8688 OR SUPPORT@CQMEDICAL.COM.

SERIOUS INCIDENTS

Please report any serious incidents (e.g. incidents which result in or have the potential to result in death or serious injury) to both CQ Medical and your country's Competent Authority.

ADDITIONAL WARNINGS

- Do not exceed pedestal manufacturer's load rating.
- Maximum patient weight of 204 kg (449 lb) or the lowest of the QUANTUM CT Overlay, kVue™ Couch and CT weight limit.
- Pinch points may exist and are properly marked on the QUANTUM CT Overlay.
- Attention is necessary when moving the Couch and CT Insert into the Gantry Bore to ensure that pinch point risk to hands and fingers are avoided. If cradle is retracted from the bore of the machine with the cradle in its lowest position, it may be possible to damage the front end eccentric cams.
- Use of this product in a manner not specified in the User Manual may be unsafe and is not recommended.
- Use of unapproved devices or attachments may be unsafe and is not recommended.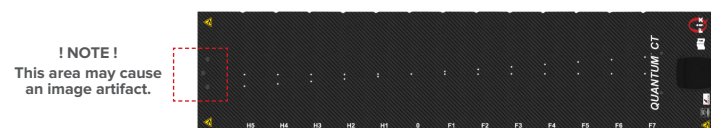

! CAUTION ! LOCATING BARS USED IN CT SCANS MAY CAUSE ARTIFACTING. MOVE LOCATING BAR OUTSIDE REGION OF INTEREST IF NECESSARY.

COLLISION HAZARDS

Attention is necessary when moving the Couch and CT Insert into the Gantry Bore to ensure that pinch point risk to hands and fingers are avoided. If cradle is retracted from the bore of the machine with the cradle in its lowest position, it may be possible to damage the front end eccentric cams.


LOAD RATING

DO NOT exceed 204 kg (449 lb) or the lowest of the QUANTUM CT Overlay and CT weight limit.

! NOTE ! When combined with the OEM couch base, the load rating is the lower of the two safe working loads. The QUANTUM CT Overlay load should NOT exceed the original couch base manufacturer's specifications.

WARNING LABELS & DESCRIPTIONS

Refer to CQmedical.com for a listing of symbols and their definitions.

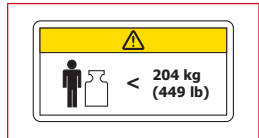	LOAD RATING DO NOT exceed 204 kg (449 lb) uniformly distributed load.
	PINCH POINTS Pinch points may exist and are indicated by this symbol.
	Hardware kit for CT is located in a cardboard box in the lid of the crate, indicated by the photo on the left.

INTENDED USE

This device is intended to immobilize, position, and reposition patients undergoing radiation therapy and simulation.

! NOTE ! United States Federal law restricts this device to sale by or on the order of a physician.

PATIENT TARGET GROUPS

Patients undergoing radiation therapy or diagnostic imaging procedures.

INTENDED USERS

The intended user for the products is a person qualified in accordance with the requirements of the regulatory region.

FEATURES

RT GATING CAMERA

Location for mounting RT Gating Camera (Fig. 1).

Fig. 1

OPERATING INSTRUCTIONS

INSTALLATION

- Place the QUANTUM CT Overlay on the top of the CT Couch centered left to right and overhanging the CT Head by 5 to 10 cm (2 to 4 inches) (Fig. 2).
- Slide the QUANTUM CT Overlay towards the CT Foot End keeping the head and foot ends near to the centerline of the CT couch (Fig. 3).
- The handle will move to the latched position automatically when the insert has been engaged properly. When only the green shows and no red, the QUANTUM CT Overlay is locked on (Fig. 4).

Fig. 2

Fig. 3

Fig. 4

REMOVAL

1. Pull/push the release handle on the QUANTUM CT Overlay, in the direction indicated by the unlock symbols, to disengage the Overlay from the CT couch (Fig. 5).
2. While activating the handle, slide the QUANTUM CT Overlay towards the head end 5 to 10 cm (2 to 4 inches) to clear the head end attachments (Fig. 6).
3. Lift off the QUANTUM CT Overlay.

Fig. 5

Fig. 6

MAINTENANCE

CLEANING THE SYSTEM

The device can be cleaned with a mild, non-abrasive cleaning or disinfecting solution. To clean, apply solution to clean cloth and wipe the surface. Visually inspect the device, if it is not clean, repeat the previous cleaning steps until visually clean. Use a clean cloth moistened with water to wipe the device to remove any cleaning agent residue. To dry, wipe the device with a clean, dry cloth. The following cleaning material has been tested and found to be appropriate for cleaning the device.

- Water
- Soap and Water

DISINFECTING THE SYSTEM

To disinfect the device surface, refer to specific instructions from the cleaning agent manufacturer.

- 10% Clorox® Bleach Solution
- Isopropyl Alcohol
- Cidex® 2.4% Activated Dialdehyde Solution

DO NOT place sharp objects on the QUANTUM CT.

Periodically check all fasteners for tightness.

CLEANING AND LUBRICATING PROCEDURE FOR CT LATCH

We recommend performing this procedure once every 3 months.

Materials List: fluoropolymer (PTFE) dry film lubricant, cotton swabs.

1. Use dry cotton swab to clean out the inside of the receiving hole (Fig. 7). The hole on the QUANTUM CT is located on the underside of the CT latch (Fig. 8).
2. Spray lubricant into the receiving holes (Fig. 9). The hole on the QUANTUM CT is located on the underside of the CT latch (Fig. 10).
3. Use dry cotton swab to wipe out loose lint and debris (Fig. 7 & 8).
4. Spray lubricant into the receiving holes again (Fig. 9 & 10).

Fig. 7

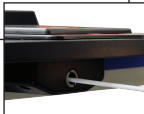
Fig. 8

Fig. 9

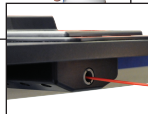
Fig. 10

PARTS LIST

1. 2000382 – Tall Eccentric
2. 2000707 – Short Eccentric (RT-4550SS2 only)
3. 2000384 – Guide Block
4. 3000118 – Black FHS 3/8-16 x 1"

kVue and Quantum are trademarks of Qfix.

Cidex is a registered trademark of Johnson & Johnson.

Clorox is a registered trademark of The Clorox Company.

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