Product Guide & User Manual

X-Trans™ Armboard

8002041

8002066

8002067

  Advena Limited Tower Business Centre, 2nd Flr Tower Street, Swatar, BKR 4013 Malta	 <i>Made in the USA by</i> Qfix 440 Church Rd, Avondale, Pennsylvania, USA
--	---

X-Trans™ is a registered trademark of QFix.
CIDEX® is a registered trademark of Johnson & Johnson.
Clorox® is a registered trademark of The Clorox Company.

General Precautions

Warning

- No modification of this equipment is allowed. If any part of this device experiences a catastrophic load, appears damaged or functions improperly, discontinue use immediately and contact CQ Medical at +1 712-737-8688 or info@CQmedical.com

Serious Incidents

Please report any serious incidents (e.g. incidents which result in or have the potential to result in death or serious injury) to both CQ Medical and your country's Competent Authority.

Load Rating

8002041 - DO NOT exceed 249 kg (550 lb) uniformly distributed load.

8002066 - DO NOT exceed 226 kg (500 lb) uniformly distributed load

8002067 - DO NOT exceed 226 kg (500 lb) uniformly distributed load

Warning Labels & Descriptions

Refer to www.CQmedical.com for a listing of symbols and their definitions.

Intended Use

This family of products is designed as radiolucent devices used for injections and/or armrests. The products are designed to be used with flat top Angiography tables and flat top imaging tables.

Note: United States Federal law restricts this device to sale by or on the order of a physician.

Patient Target Groups

Patients undergoing general medical or surgical procedures involving diagnostic imaging.

Intended Users

The intended user for the products is a person qualified in accordance with the requirements of the regulatory region.

Maintenance

Cleaning the System

The device can be cleaned with a mild, non-abrasive cleaning or disinfecting solution. To clean, apply solution to clean cloth and wipe the surface.

Disinfecting the System

The following cleaning materials have been tested and found to be appropriate for cleaning the device:

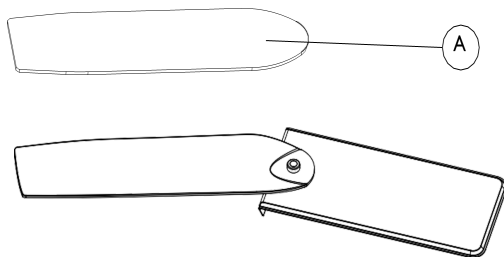
To disinfect the device surface, refer to specific instructions from the cleaning agent manufacturer.

- Water
- 10% Clorox® Bleach Solution
- Isopropyl Alcohol
- CIDEX® 2.4% Activated Dialdehyde Solution
- Soap and Water

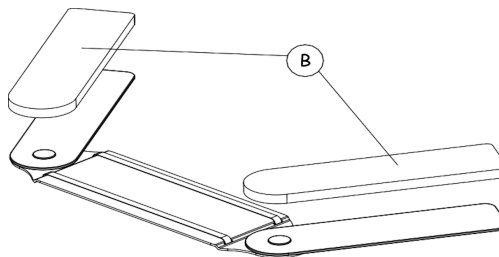
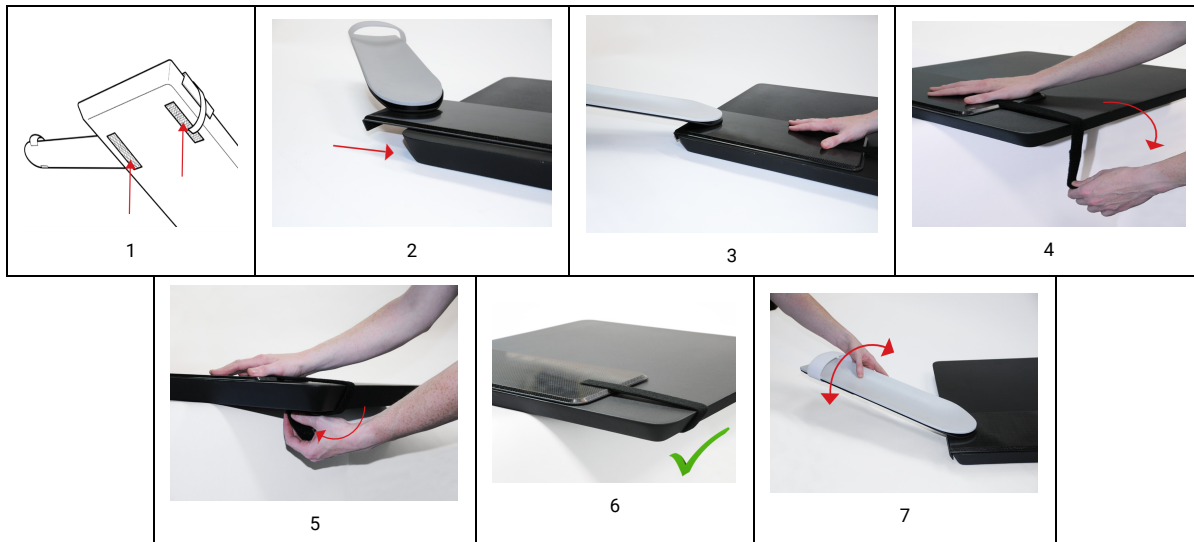
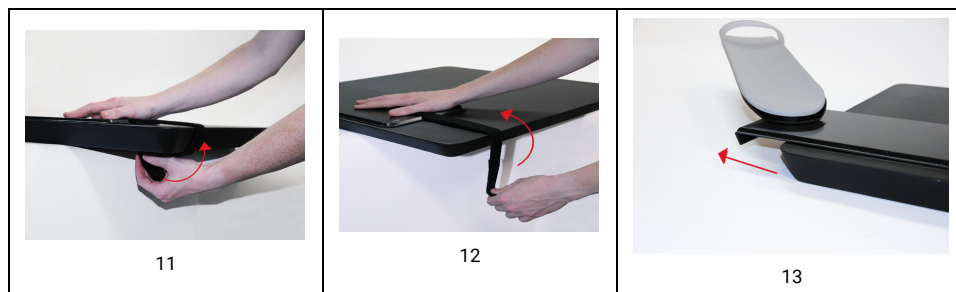
Periodically check all fasteners for tightness.

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