# PRODUCT GUIDE & USER MANUAL

RT-4700-01 Insight™ MR Overlay for Siemens®

 <b>EC REP</b> Advena Limited Tower Business Centre, 2nd Flr Tower Street, Swatar, BKR 4013 Malta	<b>UK Responsible Person</b> QServe Group UK, Ltd. 49 Greek Street Soho, London W1D 4EG United Kingdom	
	 Qfix 440 Church Rd, Avondale, Pennsylvania, USA +1 712-737-8688 www.CQmedical.com	<b>CH REP</b> Raditec Medical AG Schlossberg 5a 5454 Bellikon Switzerland

2004767\_ML\_2/ 2025-12

EN	English	NL	Dutch (Nederlands)
DE	German (Deutsch)	PL	Polish (Polski)
EL	Greek (Ελληνικά)	PT-EU	Portuguese (Português)
ES	Spanish (Español)	RO	Romanian (Română)
FI	Finnish (Suomi)	SR	Serbian (Srpski)
FR	French (Français)	SV	Swedish (Svenska)
IT	Italian (Italiano)	ZH	Chinese (简体中文)
JA	Japanese (日本語)		

EN: English

## GENERAL PRECAUTIONS

### WARNING STATEMENTS

**! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT CQ MEDICAL AT +1 712-737-8688 OR SUPPORT@CQMEDICAL.COM.**

**! NOTE !** Headphones are not compatible when using thermoplastic masks. The use of earplugs are still permitted.

### ADDITIONAL WARNINGS AND PRECAUTIONS

**! WARNING ! DO NOT CARRY A PATIENT ON THE MRI OVERLAY.**

**! WARNING ! WHEN USING FLEX LARGE COILS, ONLY USE THOSE WHICH HAVE A SPECIAL DETUNING TECHNOLOGY (PART NUMBER 10835335 OR PART NUMBER 8625761 REV 06 OR HIGHER).**

**! CAUTION ! A SCAN IN FIRST LEVEL MODE SHOULD BE AVOIDED.**

**! CAUTION ! SCANNING OF SEDATED PATIENTS SHOULD BE AVOIDED.**

**! CAUTION ! PATIENTS SHOULD BE MADE AWARE OF USING THE SQUEEZE BALL IN CASE OF HEATING SENSATIONS.**

### SERIOUS INCIDENTS

Please report any serious incidents (e.g. incidents which result in or have the potential to result in death or serious injury) to both CQ Medical and your country's Competent Authority.

## MRI SAFETY INFORMATION

**MR** Non-clinical testing has demonstrated the **Insight MR Overlay** is MR Safe. This device may be used in an MR system meeting the following conditions:

- **STATIC MAGNETIC FIELD OF 3T OR LESS.**

**! WARNING ! USE OF UNAPPROVED MR ACCESSORIES MAY RESULT IN:**

- **INJURY TO PATIENT**
- **PATIENT BURNS**

Use only proven MR Safe or MR Conditional accessories, tested and approved for your MR system.

Consider and evaluate possible distortions of the MR images used, particularly when fusing with CT datasets.

Consider the MR compatibility of accessories prior to use on your MR system.

## WARNING LABELS & DESCRIPTIONS

Refer to [CQmedical.com](http://CQmedical.com) for a listing of symbols and their definitions.

	<b>MR SAFE</b> An item that poses no known hazards resulting from exposure to any MR environment. MR Safe items are composed of materials that are electrically non-conductive, nonmetallic and nonmagnetic.
	<b>LOAD RATING</b> DO NOT exceed 243 kg (536 lb) uniformly distributed load when installed on a MR Table. When on a couch base the maximum safe working load is that of the couch base.

## INTENDED USE

The Insight™ MR Overlay is an MR Safe patient support device. It is intended to allow patients to undergo diagnostic imaging in the same position as that of the treatment position.

**! NOTE !** United States Federal law restricts this device to sale by or on the order of a physician.

### PATIENT TARGET GROUPS

Patients undergoing radiation therapy or diagnostic imaging procedures.

### INTENDED USERS

The intended user for the products is a person qualified in accordance with the requirements of the regulatory region.

## FEATURES

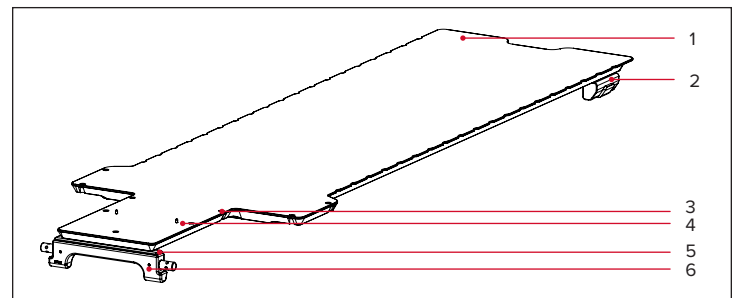
Fig. 1

## SYSTEM COMPONENTS

### 1. INSIGHT™ MR OVERLAY

The Insight MR Overlay mounts to compatible MR table tops and allows for use of patient positioning devices during MR scanning for radiotherapy.

### 2. FOOT MOUNTS

The Foot Mounts come pre-installed onto the Insight MR Overlay. The mounts secure the foot end of the Insight MR Overlay to compatible MR cradles.

### 3. MASK ATTACHMENT HOLES

The mask attachment holes on the Insight MR Overlay are compatible with head-only s-type thermoplastic masks.

### 4. SILVERMAN PINS

The Silverman pins locate Silverman headrests to the Insight MR Overlay. These may be removed when not in use.

### 5. HEAD SHIM

The Head Shim comes pre-installed on the Insight MRI Overlay. The shim provides height adjustment needed for installation of the Insight on the Head Mount.

### 6. HEAD MOUNT

The Head Mount comes pre-installed onto the Insight MR Overlay. The mount secures the head end of the Insight MR Overlay to compatible MR cradles.

## MR COMPATIBILITY CHART

MRI Scanners	Insight Overlay
Magnetom Aera	✓
Magnetom Altea	✓
Magnetom Amira	✓
Magnetom Avanto	✓
Magnetom Avanto a Tim + Dot System	✓
Magnetom Avanto Dot Upgrade	✓
Magnetom Avanto Fit	✓
Magnetom Avanto Fit Upgrade	✓
Magnetom Espree	✓
Magnetom Essenza	✓
Magnetom Essenza a Tim + Dot System	✓
Magnetom Essenza Dot Upgrade	✓
Magnetom Sola	✓
Magnetom Sola Fit	✓
Magnetom Symphony	✓
Magnetom Symphony a Tim System	✓
Biograph mMR	✓
Magnetom Lumina	✓
Magnetom Prisma	✓
Magnetom Prisma Fit Upgrade	✓
Magnetom Skyra	✓
Magnetom Skyra Fit Upgrade	✓
Magnetom Spectra	✓
Magnetom Trio	✓
Magnetom Trio a Tim System	✓
Magnetom Verio	✓
Magnetom Verio Dot Upgrade	✓
Magnetom Verio a Tim + Dot System	✓
Magnetom Vida	✓
Magnetom Vida Fit	✓
Other MRs	*

\* Not compatible with other MRI Scanners

## OPERATING INSTRUCTIONS

### INSTALLATION

1. Remove the Spine Coil Pad from your Siemens MAGNETOM® Spine Coil (Fig. 2).

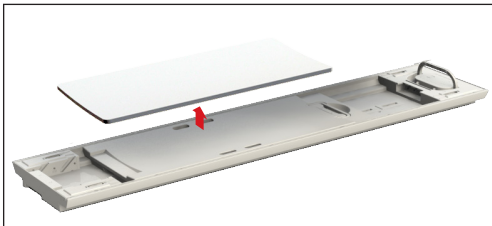
Fig. 2

2. Align Insight™ foot mounts with the Siemens Table.
3. Place MRI Overlay onto MR Table, aligning the head and foot mounts with the couch base (Fig. 3).

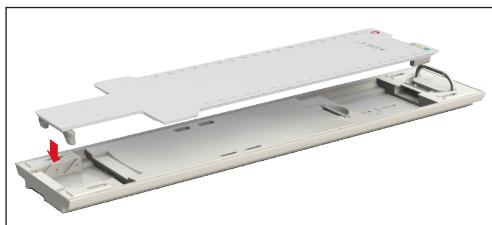
Fig. 3

4. Center table by rotating head mount knobs (Fig. 4).

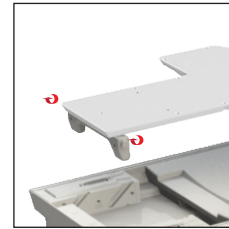
Fig. 4

### S-FRAME THERMOPLASTIC SET-UP

**! CAUTION ! EXCLUSIVELY DESIGNED FOR USE WITH S-FRAMES. THE INSIGHT MR OVERLAY IS NOT COMPATIBLE WITH OTHER THERMOPLASTIC MASK TYPES.**

1. Place desired Silverman head support onto the Insight™ MR Overlay, aligning the holes with the pins on the Insight MR Overlay.
2. Form a MOLDCARE® Cushion around patient's head and neck according to MOLDCARE IFU.
3. Place mask over patient, lining up S-Frame pins with attachment holes on the Insight MR Overlay (Fig. 5).
4. Insert pins into the Insert holes and press until each pin locks in place (Fig. 6).

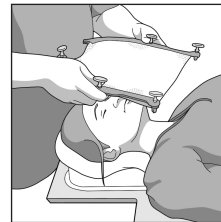
Fig. 5

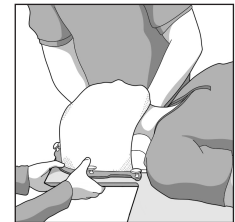
Fig. 6

### S-FRAME THERMOPLASTIC REMOVAL

1. To remove, pull up on each pin.
2. Pull up on mask and remove from patient.

### SILVERMAN PIN REMOVAL

1. Remove pins from Insight™ MR Overlay by unscrewing counter clockwise from the bottom of the device.
2. Once threads are fully disengaged, remove pins from locating holes.

### INSIGHT MR OVERLAY HEAD AND FOOT MOUNTS

1. Lift Insight MR Overlay off of MR Table.
2. Remove all M8 screws that are attaching both the Head and Foot Mounts to the Insight MR Overlay.
3. Remove all Mounts from the device.

**! NOTE ! Insight MR Overlay may be stored with Head and Foot Mounts still attached.**

### TO REPLACE THE HEAD AND FOOT MOUNTS

1. Install Foot Mounts onto the bottom side of the MRI Overlay using four (4) M8 screws (Fig. 7).
2. Install Head Mount onto the bottom side of the MRI Overlay using three (3) M8 screws (Fig. 8).

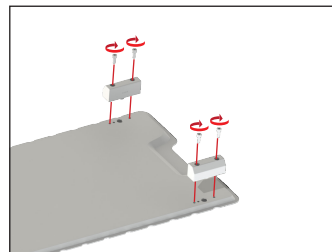
Fig. 7

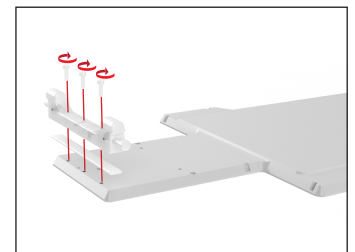
Fig. 8

## MAINTENANCE

### CLEANING THE SYSTEM

The device can be cleaned with a mild, non-abrasive cleaning or disinfecting solution. To clean, apply solution to clean cloth and wipe the surface. Visually inspect the device, if it is not clean, repeat the previous cleaning steps until visually clean. Use a clean cloth moistened with water to wipe the device to remove any cleaning agent residue. To dry, wipe the device with a clean, dry cloth. The following cleaning material has been tested and found to be appropriate for cleaning the device.


- Water
- Soap and Water

### DISINFECTING THE SYSTEM

To disinfect the device surface, refer to specific instructions from the cleaning agent manufacturer.

- 10% Clorox® Bleach Solution
- Isopropyl Alcohol
- Cidex® 2.4% Activated Dialdehyde Solution

## SPECIFICATIONS

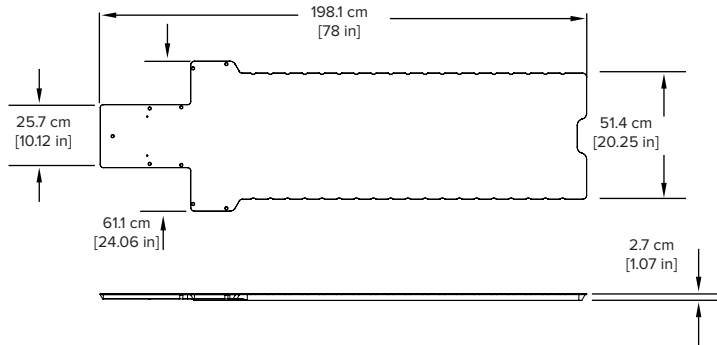## DIMENSIONS

**LENGTH (L):** 198.1 cm [78 in]

**WIDTH (W):** 61.1 cm [24.06 in]

**HEIGHT (H):** 2.7 cm [1.07 in]

## PARTS LIST

1. 2004515 – Siemens MRI Overlay Foot Mount
2. 2004517 – Siemens MRI Overlay Head Mount, MAGNETOM®
3. 3001064 – SHCS M8 x 1.25 x 20 mm
4. Hardware kit or Silverman pins

Insight is a trademark of Qfix.

Siemens®, MAGNETOM Aera®, MAGNETOM Skyra®, MAGNETOM Vida®, and MAGNETOM Sola® are registered trademarks of Siemens.

MOLDCARE® is a registered trademark of ALCARE CO, LTD.

Exact® is a registered trademark of Varian Medical Systems.

Cidex® is a registered trademark of Johnson & Johnson.

Clorox® is a registered trademark of The Clorox Company.

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## MRT-KOMPATIBILITÄTSTABELLE

MRT-Scanner	Insight Overlay
Magnetom Aera	✓
Magnetom Altea	✓
Magnetom Amira	✓
Magnetom Avanto	✓
Magnetom Avanto a Tim + Dot System	✓
Magnetom Avanto Dot Upgrade	✓
Magnetom Avanto Fit	✓
Magnetom Avanto Fit Upgrade	✓
Magnetom Espree	✓
Magnetom Essenza	✓
Magnetom Essenza a Tim + Dot System	✓
Magnetom Essenza Dot Upgrade	✓
Magnetom Sola	✓
Magnetom Sola Fit	✓
Magnetom Symphony	✓
Magnetom Symphony a Tim System	✓
Biograph mMR	✓
Magnetom Lumina	✓
Magnetom Prisma	✓
Magnetom Prisma Fit Upgrade	✓
Magnetom Skyra	✓
Magnetom Skyra Fit Upgrade	✓
Magnetom Spectra	✓
Magnetom Trio	✓
Magnetom Trio a Tim System	✓
Magnetom Verio	✓
Magnetom Verio Dot Upgrade	✓
Magnetom Verio a Tim + Dot System	✓
Magnetom Vida	✓
Magnetom Vida Fit	✓
Andere MRT-Scanner	*

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Scanners IRM	Plaque Insight
Magnetom Aera	✓
Magnetom Altea	✓
Magnetom Amira	✓
Magnetom Avanto	✓
Magnetom Avanto a Tim + Dot System	✓
Magnetom Avanto Dot Upgrade	✓
Magnetom Avanto Fit	✓
Magnetom Avanto Fit Upgrade	✓
Magnetom Espree	✓
Magnetom Essenza	✓
Magnetom Essenza a Tim + Dot System	✓
Magnetom Essenza Dot Upgrade	✓
Magnetom Sola	✓
Magnetom Sola Fit	✓
Magnetom Symphony	✓
Magnetom Symphony a Tim System	✓
Biograph mMR	✓
Magnetom Lumina	✓
Magnetom Prisma	✓
Magnetom Prisma Fit Upgrade	✓
Magnetom Skyra	✓
Magnetom Skyra Fit Upgrade	✓
Magnetom Spectra	✓
Magnetom Trio	✓
Magnetom Trio a Tim System	✓
Magnetom Verio	✓
Magnetom Verio Dot Upgrade	✓
Magnetom Verio a Tim + Dot System	✓
Magnetom Vida	✓
Magnetom Vida Fit	✓
Autres scanners IRM	*

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Scanner per RM	Materassino Insight
Magnetom Aera	✓
Magnetom Altea	✓
Magnetom Amira	✓
Magnetom Avanto	✓
Magnetom Avanto a Tim + Dot System	✓
Magnetom Avanto Dot Upgrade	✓
Magnetom Avanto Fit	✓
Magnetom Avanto Fit Upgrade	✓
Magnetom Espree	✓
Magnetom Essenza	✓
Magnetom Essenza a Tim + Dot System	✓
Magnetom Essenza Dot Upgrade	✓
Magnetom Sola	✓
Magnetom Sola Fit	✓
Magnetom Symphony	✓
Magnetom Symphony a Tim System	✓
Biograph mMR	✓
Magnetom Lumina	✓
Magnetom Prisma	✓
Magnetom Prisma Fit Upgrade	✓
Magnetom Skyra	✓
Magnetom Skyra Fit Upgrade	✓
Magnetom Spectra	✓
Magnetom Trio	✓
Magnetom Trio a Tim System	✓
Magnetom Verio	✓
Magnetom Verio Dot Upgrade	✓
Magnetom Verio a Tim + Dot System	✓
Magnetom Vida	✓
Magnetom Vida Fit	✓
Altri scanner per RM	*

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MRIスキャナ	Insightオーバーレイ
Magnetom Aera	✓
Magnetom Altea	✓
Magnetom Amira	✓
Magnetom Avanto	✓
Magnetom Avanto a Tim + Dot System	✓
Magnetom Avanto Dot Upgrade	✓
Magnetom Avanto Fit	✓
Magnetom Avanto Fit Upgrade	✓
Magnetom Espree	✓
Magnetom Essenza	✓
Magnetom Essenza a Tim + Dot System	✓
Magnetom Essenza Dot Upgrade	✓
Magnetom Sola	✓
Magnetom Sola Fit	✓
Magnetom Symphony	✓
Magnetom Symphony a Tim System	✓
Biograph mMR	✓
Magnetom Lumina	✓
Magnetom Prisma	✓
Magnetom Prisma Fit Upgrade	✓
Magnetom Skyra	✓
Magnetom Skyra Fit Upgrade	✓
Magnetom Spectra	✓
Magnetom Trio	✓
Magnetom Trio a Tim System	✓
Magnetom Verio	✓
Magnetom Verio Dot Upgrade	✓
Magnetom Verio a Tim + Dot System	✓
Magnetom Vida	✓
Magnetom Vida Fit	✓
他のMR	*

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## MR-COMPATIBILITEITSTABEL

MRI-scanners	Insight-overlay
Magnetom Aera	✓
Magnetom Altea	✓
Magnetom Amira	✓
Magnetom Avanto	✓
Magnetom Avanto a Tim + Dot System	✓
Magnetom Avanto Dot Upgrade	✓
Magnetom Avanto Fit	✓
Magnetom Avanto Fit Upgrade	✓
Magnetom Espree	✓
Magnetom Essenza	✓
Magnetom Essenza a Tim + Dot System	✓
Magnetom Essenza Dot Upgrade	✓
Magnetom Sola	✓
Magnetom Sola Fit	✓
Magnetom Symphony	✓
Magnetom Symphony a Tim System	✓
Biograph mMR	✓
Magnetom Lumina	✓
Magnetom Prisma	✓
Magnetom Prisma Fit Upgrade	✓
Magnetom Skyra	✓
Magnetom Skyra Fit Upgrade	✓
Magnetom Spectra	✓
Magnetom Trio	✓
Magnetom Trio a Tim System	✓
Magnetom Verio	✓
Magnetom Verio Dot Upgrade	✓
Magnetom Verio a Tim + Dot System	✓
Magnetom Vida	✓
Magnetom Vida Fit	✓
Andere MR's	*

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Skanery RM	Nakładka Insight
Magnetom Aera	✓
Magnetom Altea	✓
Magnetom Amira	✓
Magnetom Avanto	✓
Magnetom Avanto a Tim + Dot System	✓
Magnetom Avanto Dot Upgrade	✓
Magnetom Avanto Fit	✓
Magnetom Avanto Fit Upgrade	✓
Magnetom Espree	✓
Magnetom Essenza	✓
Magnetom Essenza a Tim + Dot System	✓
Magnetom Essenza Dot Upgrade	✓
Magnetom Sola	✓
Magnetom Sola Fit	✓
Magnetom Symphony	✓
Magnetom Symphony a Tim System	✓
Biograph mMR	✓
Magnetom Lumina	✓
Magnetom Prisma	✓
Magnetom Prisma Fit Upgrade	✓
Magnetom Skyra	✓
Magnetom Skyra Fit Upgrade	✓
Magnetom Spectra	✓
Magnetom Trio	✓
Magnetom Trio a Tim System	✓
Magnetom Verio	✓
Magnetom Verio Dot Upgrade	✓
Magnetom Verio a Tim + Dot System	✓
Magnetom Vida	✓
Magnetom Vida Fit	✓
Inne systemy RM	*

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## TABELA DE COMPATIBILIDADE COM RM

Aparelhos de RM	Sobrecamada Insight
Magnetom Aera	✓
Magnetom Altea	✓
Magnetom Amira	✓
Magnetom Avanto	✓
Magnetom Avanto a Tim + Dot System	✓
Magnetom Avanto Dot Upgrade	✓
Magnetom Avanto Fit	✓
Magnetom Avanto Fit Upgrade	✓
Magnetom Espree	✓
Magnetom Essenza	✓
Magnetom Essenza a Tim + Dot System	✓
Magnetom Essenza Dot Upgrade	✓
Magnetom Sola	✓
Magnetom Sola Fit	✓
Magnetom Symphony	✓
Magnetom Symphony a Tim System	✓
Biograph mMR	✓
Magnetom Lumina	✓
Magnetom Prisma	✓
Magnetom Prisma Fit Upgrade	✓
Magnetom Skyra	✓
Magnetom Skyra Fit Upgrade	✓
Magnetom Spectra	✓
Magnetom Trio	✓
Magnetom Trio a Tim System	✓
Magnetom Verio	✓
Magnetom Verio Dot Upgrade	✓
Magnetom Verio a Tim + Dot System	✓
Magnetom Vida	✓
Magnetom Vida Fit	✓
Other MRs	*

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Scanere IRM	Strat de acoperire Insight
Magnetom Aera	✓
Magnetom Altea	✓
Magnetom Amira	✓
Magnetom Avanto	✓
Magnetom Avanto a Tim + Dot System	✓
Magnetom Avanto Dot Upgrade	✓
Magnetom Avanto Fit	✓
Magnetom Avanto Fit Upgrade	✓
Magnetom Espree	✓
Magnetom Essenza	✓
Magnetom Essenza a Tim + Dot System	✓
Magnetom Essenza Dot Upgrade	✓
Magnetom Sola	✓
Magnetom Sola Fit	✓
Magnetom Symphony	✓
Magnetom Symphony a Tim System	✓
Biograph mMR	✓
Magnetom Lumina	✓
Magnetom Prisma	✓
Magnetom Prisma Fit Upgrade	✓
Magnetom Skyra	✓
Magnetom Skyra Fit Upgrade	✓
Magnetom Spectra	✓
Magnetom Trio	✓
Magnetom Trio a Tim System	✓
Magnetom Verio	✓
Magnetom Verio Dot Upgrade	✓
Magnetom Verio a Tim + Dot System	✓
Magnetom Vida	✓
Magnetom Vida Fit	✓
Alte sisteme RM	*

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## TABELA KOMPATIBILNOSTI

Tip aparata	Insight ploča
Magnetom Aera	✓
Magnetom Altea	✓
Magnetom Amira	✓
Magnetom Avanto	✓
Magnetom Avanto a Tim + Dot System	✓
Magnetom Avanto Dot Upgrade	✓
Magnetom Avanto Fit	✓
Magnetom Avanto Fit Upgrade	✓
Magnetom Espree	✓
Magnetom Essenza	✓
Magnetom Essenza a Tim + Dot System	✓
Magnetom Essenza Dot Upgrade	✓
Magnetom Sola	✓
Magnetom Sola Fit	✓
Magnetom Symphony	✓
Magnetom Symphony a Tim System	✓
Biograph mMR	✓
Magnetom Lumina	✓
Magnetom Prisma	✓
Magnetom Prisma Fit Upgrade	✓
Magnetom Skyra	✓
Magnetom Skyra Fit Upgrade	✓
Magnetom Spectra	✓
Magnetom Trio	✓
Magnetom Trio a Tim System	✓
Magnetom Verio	✓
Magnetom Verio Dot Upgrade	✓
Magnetom Verio a Tim + Dot System	✓
Magnetom Vida	✓
Magnetom Vida Fit	✓
Ostali modeli	*

\* Nije kompatibilna sa ostalim modelima aparata za snimanje magnetnom rezonancom

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## MR-KOMPATIBILITETSTABELL

MRT-kameror	Insight-överlägg
Magnetom Aera	✓
Magnetom Altea	✓
Magnetom Amira	✓
Magnetom Avanto	✓
Magnetom Avanto a Tim + Dot System	✓
Magnetom Avanto Dot Upgrade	✓
Magnetom Avanto Fit	✓
Magnetom Avanto Fit Upgrade	✓
Magnetom Espree	✓
Magnetom Essenza	✓
Magnetom Essenza a Tim + Dot System	✓
Magnetom Essenza Dot Upgrade	✓
Magnetom Sola	✓
Magnetom Sola Fit	✓
Magnetom Symphony	✓
Magnetom Symphony a Tim System	✓
Biograph mMR	✓
Magnetom Lumina	✓
Magnetom Prisma	✓
Magnetom Prisma Fit Upgrade	✓
Magnetom Skyra	✓
Magnetom Skyra Fit Upgrade	✓
Magnetom Spectra	✓
Magnetom Trio	✓
Magnetom Trio a Tim System	✓
Magnetom Verio	✓
Magnetom Verio Dot Upgrade	✓
Magnetom Verio a Tim + Dot System	✓
Magnetom Vida	✓
Magnetom Vida Fit	✓
Andra MR	*

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MRI 扫描仪	Insight 覆盖罩
Magnetom Aera	✓
Magnetom Altea	✓
Magnetom Amira	✓
Magnetom Avanto	✓
Magnetom Avanto a Tim + Dot System	✓
Magnetom Avanto Dot Upgrade	✓
Magnetom Avanto Fit	✓
Magnetom Avanto Fit Upgrade	✓
Magnetom Espree	✓
Magnetom Essenza	✓
Magnetom Essenza a Tim + Dot System	✓
Magnetom Essenza Dot Upgrade	✓
Magnetom Sola	✓
Magnetom Sola Fit	✓
Magnetom Symphony	✓
Magnetom Symphony a Tim System	✓
Biograph mMR	✓
Magnetom Lumina	✓
Magnetom Prisma	✓
Magnetom Prisma Fit Upgrade	✓
Magnetom Skyra	✓
Magnetom Skyra Fit Upgrade	✓
Magnetom Spectra	✓
Magnetom Trio	✓
Magnetom Trio a Tim System	✓
Magnetom Verio	✓
Magnetom Verio Dot Upgrade	✓
Magnetom Verio a Tim + Dot System	✓
Magnetom Vida	✓
Magnetom Vida Fit	✓
其他 MR	*

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