



positioning  
patients for life.®

# PRODUCT GUIDE & USER MANUAL

RT-4488-01, RT-4488-02, RT-4488-03,  
RT-4488-04, RT-4488-05, & RT-4488-06

SofTouch™ Couch Pad



# GENERAL PRECAUTIONS

## WARNING STATEMENTS

**! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT QFIX AT +1 484-720-6054 OR TECHSUPPORT@QFIX.COM.**

## WARNING LABELS & DESCRIPTIONS

Refer to [Qfix.com](http://Qfix.com) for a listing of symbols and their definitions.



### CONSULT INSTRUCTIONS FOR USE

The SofTouch™ Couch Pad may cover important warning labels, precautions and labeling. Refer to the IFU for the device the Couch Pad is being used upon for these warnings, precautions and additional labeling.

# MAINTENANCE


The device can be cleaned with a mild, non-abrasive cleaning or disinfecting solution. To clean, apply solution to clean cloth and wipe the surface.

The following cleaning materials have been tested and found to be appropriate for cleaning the surface. To disinfect the the surface, refer to specific instructions from the cleaning agent manufacturer.

- Water
- 10% Clorox® Bleach Solution
- Isopropyl Alcohol
- Cidex® 2.4% Activated Dialdehyde Solution
- Soap and water

DO NOT spray directly onto the Couch Pad.

DO NOT place sharp objects on the Couch Pad.

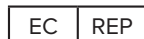

Advena Limited  
Tower Business Centre, 2nd Flr  
Tower Street, Swatar, BKR 4013  
Malta



Made in the USA by  
Qfix  
440 Church Rd, Avondale, Pennsylvania, USA  
+1 610-268-0585 [www.Qfix.com](http://www.Qfix.com)

SofTouch is a trademark of Qfix.

Cidex® is a registered trademark of Johnson & Johnson

Clorox® is a registered trademark of The Clorox Company.

2004784\_EN\_E