



positioning
patients for life.®

PRODUCT GUIDE & USER MANUAL

8002215

Access™ Prone Pad Kit



EC	REP	Advena Limited Tower Business Centre, 2nd Flr Tower Street, Swatar, BKR 4013 Malta	 Made in the USA by Qfix 440 Church Rd, Avondale, Pennsylvania, USA +1 610-268-0585 www.Qfix.com
----	-----	---	--

Access is a trademark of Qfix.

Cidex® is a registered trademark of Johnson & Johnson.

Clorox® is a registered trademark of The Clorox Company.

TABLE OF CONTENTS

GENERAL PRECAUTIONS.....	4
WARNING STATEMENTS	4
PATIENT TARGET GROUPS	4
INTENDED USERS	4
SERIOUS INCIDENTS.....	4
WARNING LABELS & DESCRIPTIONS	4
OPERATING INSTRUCTIONS.....	5
SET-UP	5
MAINTENANCE.....	7
CLEANING THE SYSTEM	7
DISINFECTING THE SYSTEM.....	7

GENERAL PRECAUTIONS

WARNING STATEMENTS

! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT QFIX AT +1 484-720-6054 OR TECHSUPPORT@QFIX.COM.

PATIENT TARGET GROUPS

Patients undergoing radiation therapy or diagnostic imaging procedures.

INTENDED USERS

The intended user for the products is a person qualified in accordance with the requirements of the regulatory region.

SERIOUS INCIDENTS

Please report any serious incidents (e.g. incidents which result in or have the potential to result in death or serious injury) to both Qfix and your country's Competent Authority.

WARNING LABELS & DESCRIPTIONS

Refer to Qfix.com for a listing of symbols and their definitions.

OPERATING INSTRUCTIONS

SET-UP

ATTACHING LEG COMPONENT PAD

1. Remove the protective film on the underside of the pad to expose the adhesive backing of the hook and loop strips. (Fig. 1)
2. Align Pad with the foot end and sides of the Access™ Prone Leg Component and adhere. Press firmly on the areas where the hook and loop strips are located. (Fig. 2)

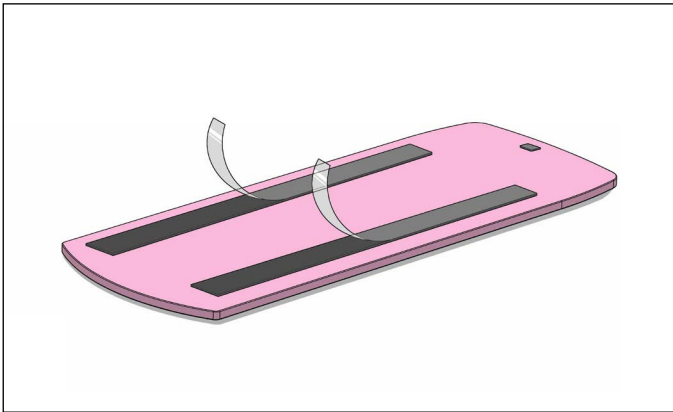


Fig. 1

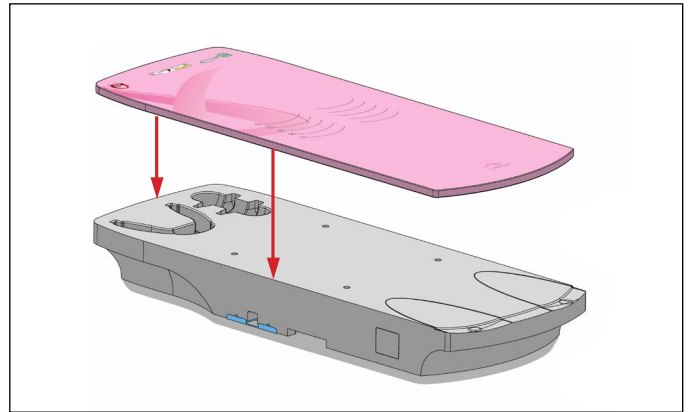


Fig. 2

ATTACHING INSERT PAD

1. Remove the protective film on the underside of the pad to expose the adhesive backing of the hook and loop strips. (Fig. 1)
2. Align Pad with the breast cut-out of the Access™ Prone Insert and adhere. Press firmly on the areas where the hook and loop strips are located. (Fig. 2)

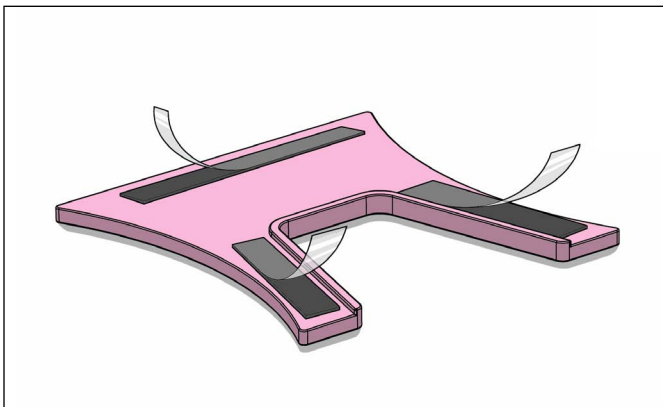


Fig. 3

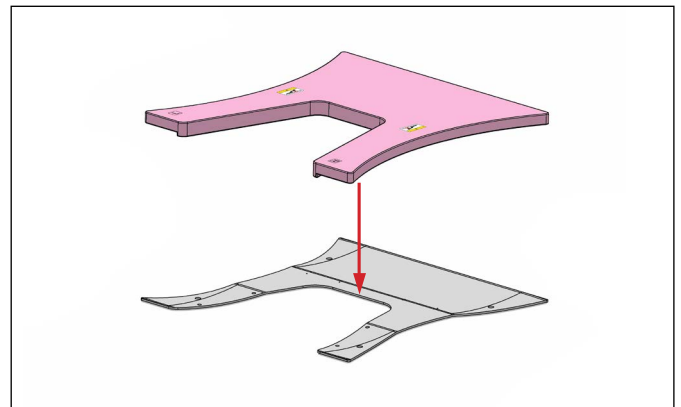


Fig. 4

OPERATING INSTRUCTIONS

SET-UP

ATTACHING HEAD COMPONENT PADS

1. Remove the protective film on the underside of the pad to expose the adhesive backing of the hook and loop squares. (Fig. 5)
2. Align Pads with the head end and handle holes of the Access™ Prone Head Component and adhere. Press firmly on the areas where the hook and loop squares are located. (Fig. 6)

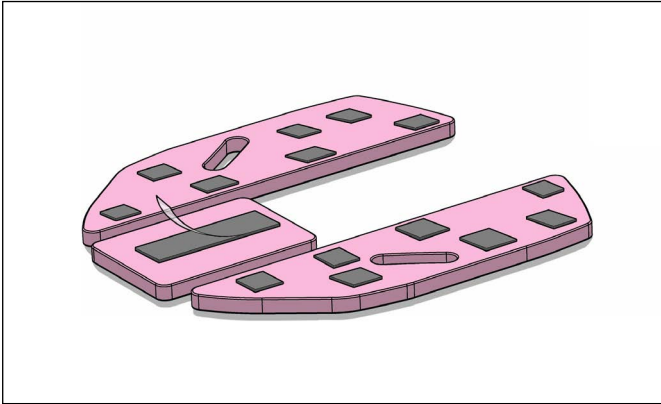


Fig. 5

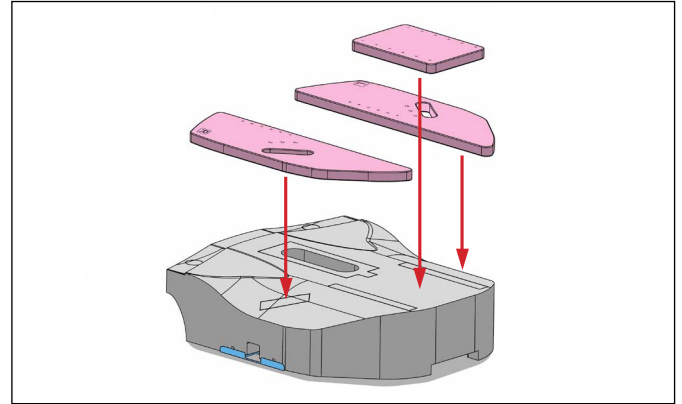


Fig. 6

MAINTENANCE

CLEANING THE SYSTEM

The Access™ Prone Pad can be cleaned with a mild, non-abrasive cleaning or disinfecting solution. To clean, apply solution to clean cloth and wipe the surface.

DISINFECTING THE SYSTEM

The following cleaning materials have been tested and found to be appropriate for cleaning the device surface. To disinfect the surface, refer to specific instructions from the cleaning agent manufacturer.

- Water
- A 10% Clorox® Bleach Solution
- Isopropyl Alcohol
- Cidex® 2.4% Activated Dialdehyde Solution
- Soap and Water

DO NOT spray directly onto the Access™ Prone Pad.

DO NOT place sharp objects on the Access™ Prone Pad.



440 Church Road
Avondale, PA 19311 USA
www.Qfix.com

+1 610.268.0585 / 800.526.5247

+1 610.268.0588 / 800.831.8174

sales@Qfix.com