


REPLACEMENT INSTRUCTIONS

Neuro Head Holder

INCLUDED MATERIALS

- Updated channel
- Drop stop
- Plastic screw
- Wing nut

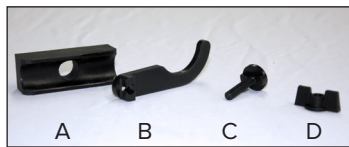

1. Remove straps and pad (Fig. 1 & Fig. 2).

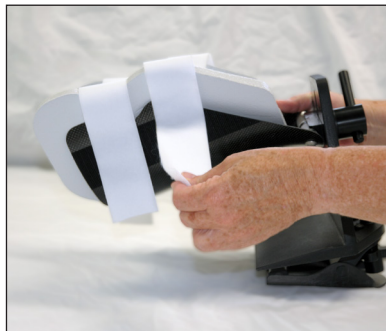


Fig. 1



Fig. 2

2. Loosen knob by turning counterclockwise (Fig. 3).
3. Remove knob and black and yellow washers from the headshell (Fig. 4). Discard black washer (Fig. 5). Keep knob and yellow washer (Fig. 6).



Fig. 3



Fig. 4

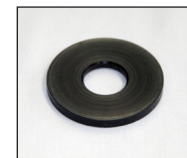


Fig. 5



Fig. 6

4. Remove headshell from base (Fig. 7).
5. Remove black channel from headshell and discard (Fig. 8).
6. Replace with supplied, updated channel (Fig. 9).

7. Place the updated channel onto the head shell (Fig. 10).

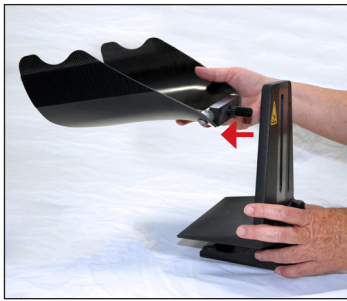


Fig. 7

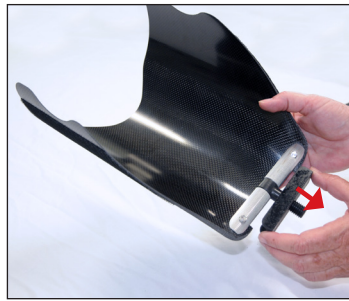


Fig. 8

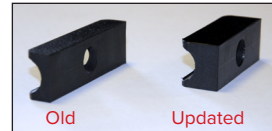


Fig. 9

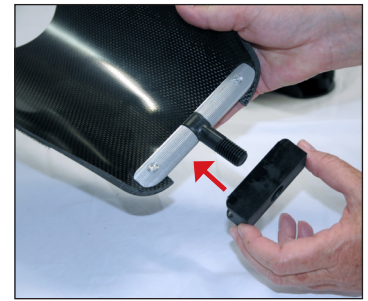


Fig. 10

8. Insert head shell/channel back into base (Fig. 11) and reattach using the original yellow washer (Fig. 12) and knob. Turn the knob clockwise to tighten (Fig. 13).



Fig. 11



Fig. 12



Fig. 13

9. Attach drop stop into narrow slot on same side as knob (Fig 14).
10. Place screw into drop stop from same side as head shell (Fig 15).
11. Place wing nut onto screw and tighten into place by turning clockwise (Fig. 16).
12. Tighten knob. Loosen wing nut and adjust placement of drop stop to contact knob (Fig. 17). Tighten wing nut to secure placement.



Fig. 14

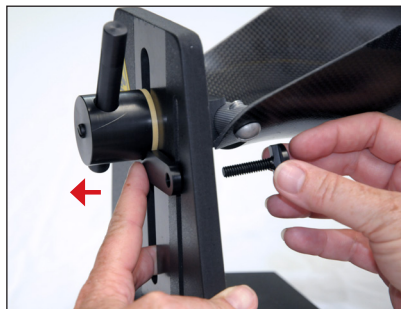


Fig. 15



Fig. 16



Fig. 17

13. Replace the pads and straps.