




positioning
patients for life.®

INSTALLATION MANUAL

RT-4550KVPBRPM & RT-4550KVPBRPM2

kVue™ CT Phantom Mount &

kVue™ CT Phantom Mount Generation 2

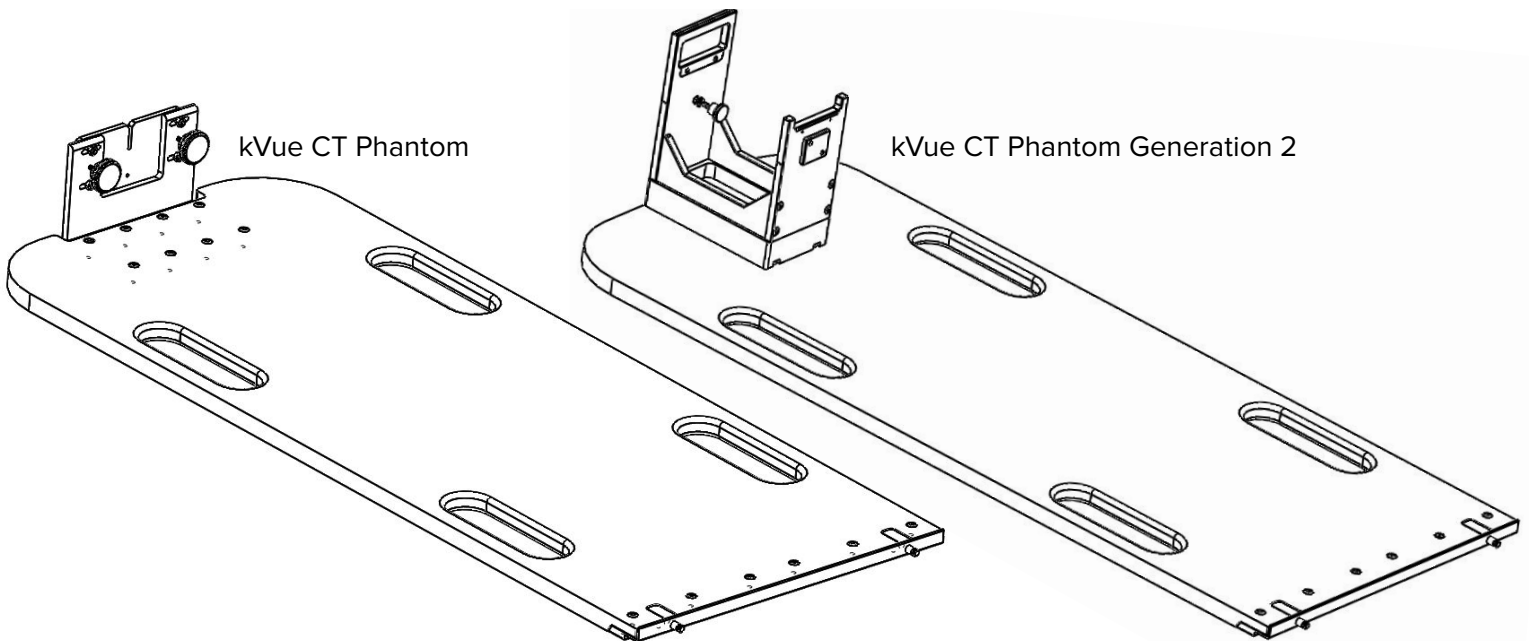



EC REP	Advena Limited Tower Business Centre, 2nd Flr Tower Street, Swatar, BKR 4013 Malta	Made in the USA by Qfix 440 Church Rd, Avondale, Pennsylvania, USA +1 484-720-6053 www.Qfix.com
------------------------------	---	--

kVue is a trademark of Qfix.

TABLE OF CONTENTS

GENERAL PRECAUTIONS..... 4

 WARNING STATEMENTS 4

 WARNING LABELS & DESCRIPTIONS 4

KVUE CT PHANTOM MOUNT..... 5

 PARTS PROVIDED..... 6

 INSTALLATION INSTRUCTIONS FOR PHILLIPS MACHINES7

 INSTALLATION INSTRUCTIONS FOR SIEMENS MACHINES..... 8

KVUE CT PHANTOM MOUNT GENERATION 2 9

 PARTS PROVIDED.....10

 INSTALLATION INSTRUCTIONS FOR PHILLIPS MACHINES 11

GENERAL PRECAUTIONS

WARNING STATEMENTS

! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT QFIX AT +1 484-720-6053 OR TECHSUPPORT@QFIX.COM.

WARNING LABELS & DESCRIPTIONS


Refer to Qfix.com for a listing of symbols and their definitions.

kVue CT Phantom Mount

INSTALLATION MANUAL

INSTALLATION

PARTS PROVIDED

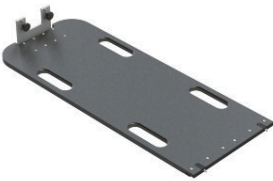 (1) kVue CT Phantom Mount	 (1) Phillips, kVue Phantom Holder Plate (2004063)
 (1) Siemens, kVue Phantom Holder Plate (2006959)	TOOLS REQUIRED (not provided): #3 Drive Cross-head Screw Driver 7/16" Wrench or Adjustable Wrench

INSTALLATION

INSTALLATION INSTRUCTIONS FOR PHILLIPS MACHINES

If installing on a Phillips machine, use the kVue CT Phantom Mount as is out of the packaging.
The kVue CT Phantom Mount comes with the Phillips Phantom Holder Plate (2004063) pre-installed.

INSTALLATION

INSTALLATION INSTRUCTIONS FOR SIEMENS MACHINES

1. If installing on a Siemens machine, unscrew the 2 plastic knobs (Fig. 1).
2. Remove the 4 screws and the 4 nuts using the #3 drive cross-head screw driver and $\frac{7}{16}$ " wrench (Fig. 2).
3. Remove the Phillips Phantom Holder Plate (2004063) and replace with the Siemens Phantom Holder Plate (2006959) ensuring that the chamfered edge of the plate is facing inward (Fig. 3 & 4).
4. Replace the 4 screws and 4 nuts using the #3 drive cross-head screw driver and $\frac{7}{16}$ " wrench (Fig. 5).
5. Replace the 2 plastic knobs (Fig. 6).

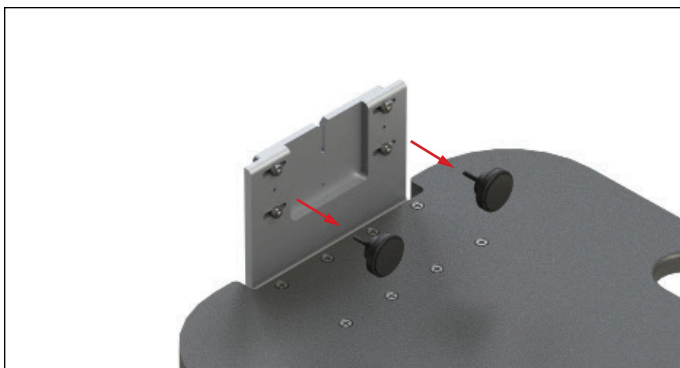


Fig. 1

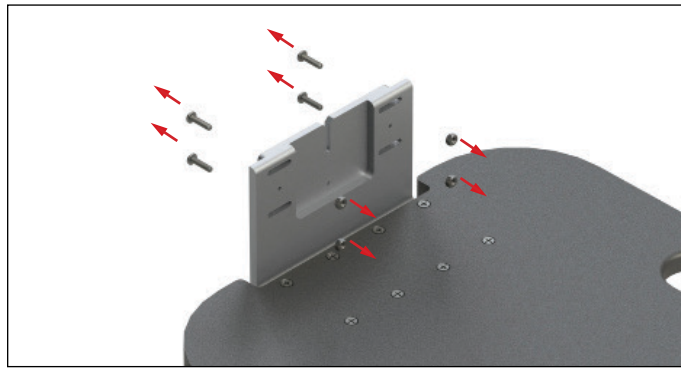


Fig. 2

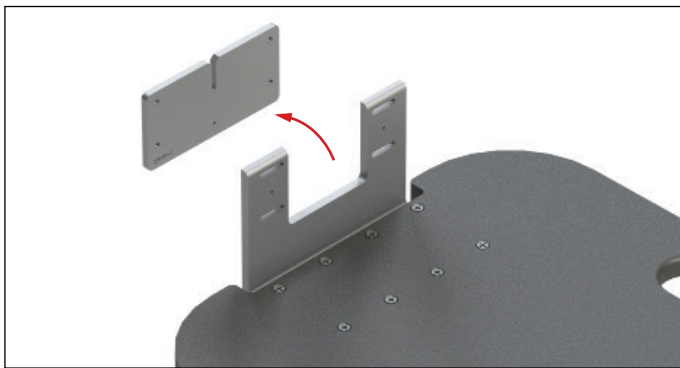


Fig. 3

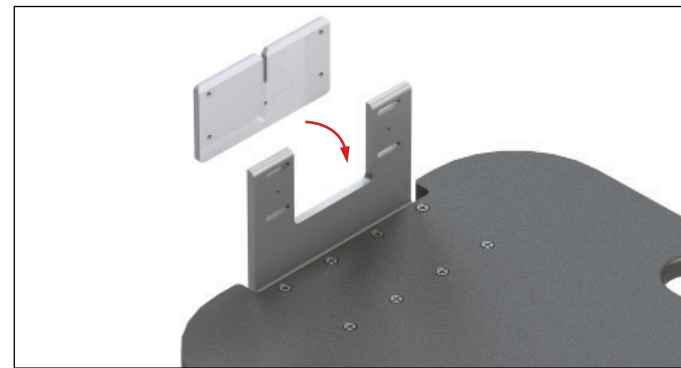


Fig. 4

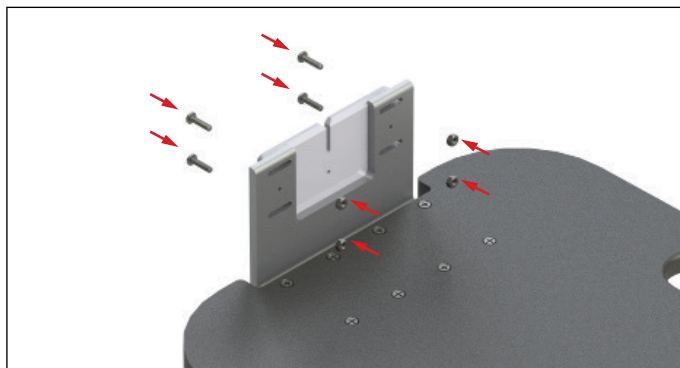


Fig. 5


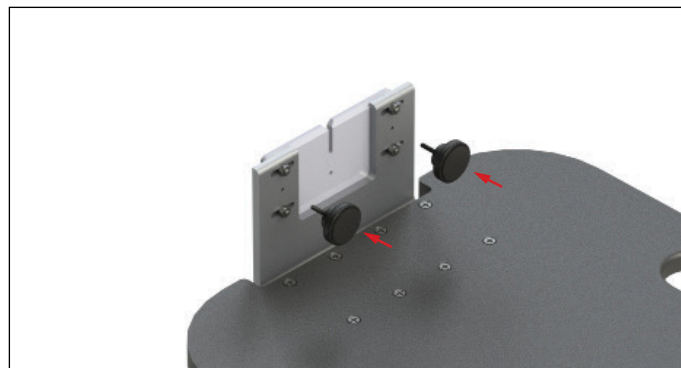


Fig. 6


kVue CT Phantom Mount Generation 2 INSTALLATION MANUAL

INSTALLATION

PARTS PROVIDED

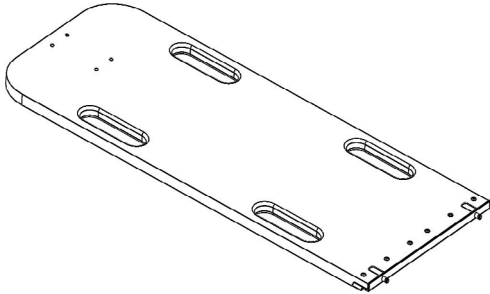

(1) kVue CT Phantom Holder Base Assembly

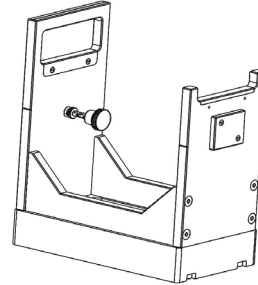

(1) Phantom Holder Top



(4) $\frac{3}{8}$ – 16 x 1.375" Flat Head Screws

TOOLS REQUIRED (not provided):

$\frac{7}{32}$ " Allen Wrench

INSTALLATION

INSTALLATION INSTRUCTIONS FOR PHILLIPS MACHINES

1. Line up the 4 holes on the bottom of the Phantom Holder Top with the four holes on the head end of the kVue CT Phantom Holder Base Assembly (Fig. 7). Then, place the Phantom Holder Top on the kVue CT Phantom Holder Base Assembly with the holes still aligned (Fig. 7).

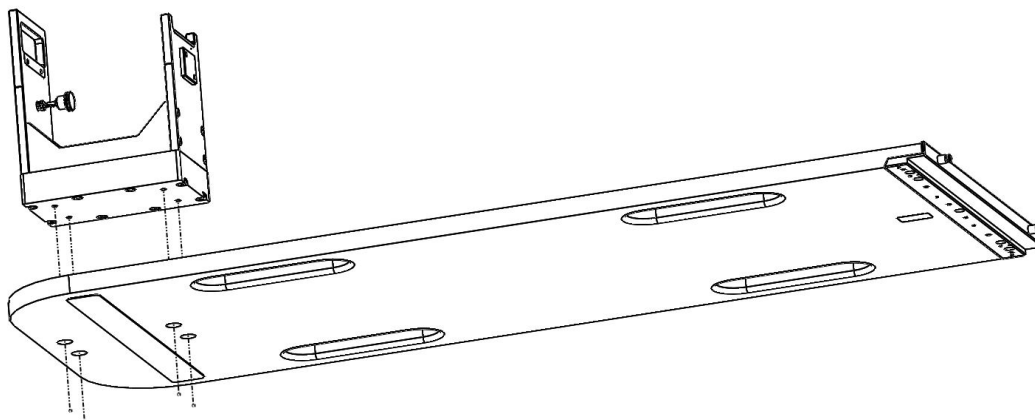


Fig. 7

2. Use four $\frac{3}{8}$ – 16 x 1.375" flat head screws to attach the Phantom Holder Top onto the kVue CT Phantom Holder Base Assembly from the bottom (Fig. 8 & 9).

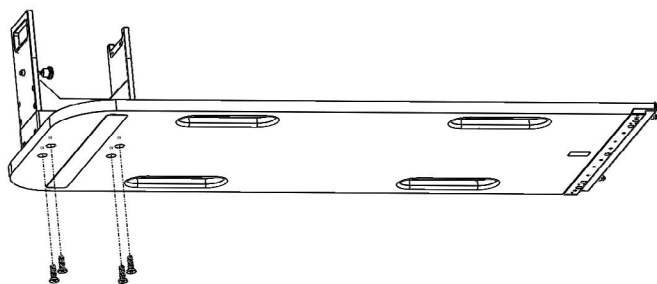


Fig. 8

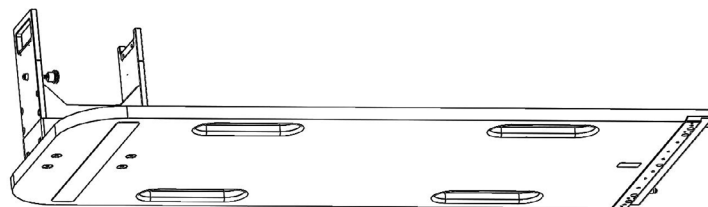


Fig. 9



440 Church Road
Avondale, PA 19311 USA
www.Qfix.com

+1 484.720.6053 / 800.526.5247

+1 610.268.0588 / 800.831.8174

sales@Qfix.com