RT-4700-02

RT-4700-03

PRODUCT GUIDE & USER MANUAL

RT-4700-02 Insight™ MR Coil Holders
RT-4700-03 for Siemens®

	EC REP Advena Limited Tower Business Centre, 2nd Flr Tower Street, Swatar, BKR 4013 Malta	UK Responsible Person QServe Group UK, Ltd. 49 Greek Street Soho, London W1D 4EG United Kingdom	
	 Made in the USA by Qfix 440 Church Rd, Avondale, Pennsylvania, USA +1 484-720-6053 www.Qfix.com	CH REP	Raditec Medical AG Schlossberg 5a 5454 Bellikon Switzerland

2007254_ML_1 / 2024-05

EN	English	JA	Japanese (日本語)
DE	German (Deutsch)	NL	Dutch (Nederlands)
ES	Spanish (Español)	RO	Romanian (Română)
FI	Finnish (Suomi)	SR	Serbian (Srpski)
FR	French (Français)	SV	Swedish (Svenska)
IT	Italian (Italiano)	ZH	Chinese (简体中文)

EN: English

GENERAL PRECAUTIONS

WARNING STATEMENTS

! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT QFIX AT +1 484-720-6053 OR TECHSUPPORT@QFIX.COM.

SERIOUS INCIDENTS

Please report any serious incidents (e.g. incidents which result in or have the potential to result in death or serious injury) to both Qfix and your country's Competent Authority.

SAFETY INFORMATION

In order to operate the Insight Coil Holders accurately and safely, the operating personnel must have the necessary expertise in a hospital setting.

To ensure safe use of the Insight Coil Holders, it is recommended that the user is educated and trained on the safe operation of the system prior to use.

MRI SAFETY INFORMATION

Consider and evaluate possible distortions of the MR images acquired. Establish a quality process in order to assess remaining distortions independent of system type, size of Field of View, and MR measurement parameters.

MR Non-clinical testing has demonstrated the **Insight Coil Holders** are MR Safe. These devices may be used in an MR system meeting the following conditions.

- **STATIC MAGNETIC FIELD OF 3T OR LESS.**
- **WITH DEVICES LISTED IN THE HEAD COIL HOLDER COMPATIBILITY CHART**

! WARNING ! USE OF UNAPPROVED MR ACCESSORIES MAY RESULT IN:

- **INJURY TO PATIENT**
- **PATIENT BURNS**

Use only proven MR Safe or MR Conditional accessories, tested and approved for your MR system.

Consider the MR compatibility of accessories prior to use on your MR system.

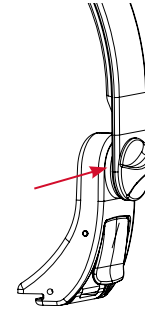
ADDITIONAL PRECAUTIONS

The use of additional devices may create additional trapping zones not described in this IFU. Inspect product for damage before each use.

! WARNING ! OVER LOOSENING THE PIVOT KNOB PAST THE STOP MAY DAMAGE THE PIVOT MECHANISM.

TRAPPING ZONES

! WARNING ! TRAPPING ZONE IS CREATED BETWEEN BRIDGE AND CLAMP WHEN KNOB IS UNSCREWED.

WARNING LABELS & DESCRIPTIONS

Refer to Qfix.com for a listing of symbols and their definitions.

MR SAFE

An item that poses no known hazards resulting from exposure to any MR environment. MR Safe items are composed of materials that are electrically non-conductive, nonmetallic and nonmagnetic.

INTENDED USE

The Insight Coil Holders are MR Safe devices intended to position body and head coils to allow the patient to undergo diagnostic imaging in the same position as that of the treatment position.

! NOTE ! United States Federal law restricts this device to sale by or on the order of a physician.

PATIENT TARGET GROUPS

Patients undergoing radiation therapy or diagnostic imaging procedures.

INTENDED USERS

The intended user for the products is a person qualified in accordance with the requirements of the regulatory region.

RT-4700-02 INSIGHT MR HEAD COIL HOLDER

SYSTEM COMPONENTS

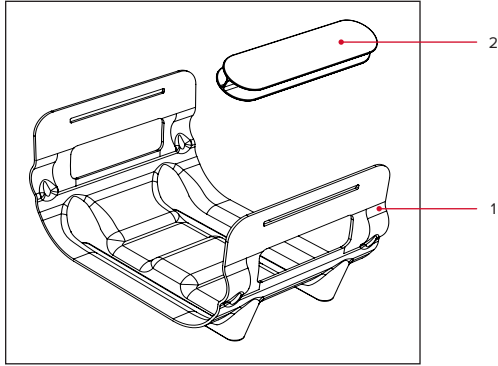
Fig. 1

1. Head Coil Holder
2. Coil Connector

HEAD COIL HOLDER COMPATIBILITY CHART

Coil	Head Coil Holder
MAGNETOM Aera® 1.5T Flex4 Large (x2) PN: 10185552	✓
MAGNETOM Skyra® and MAGNETOM Vida® 3T Flex4 Large (x2) PN: 10835335	✓ *
MAGNETOM Skyra 3T Flex4 Large (x2) PN: 8625761 Rev 06 or higher	✓ *
Other Coils	**

* Only use Flex4 Large Coils which have special detuning technology (specified above). If Flex4 Large coils other than specified are used in a setup per the Installation section, it may cause higher SAR values in the patient than the scanner is configured to monitor. For new coil versions contact Siemens sales.

** Qfix does not recommend the use of other coils with the Head Coil Holder.

INSIGHT MR HEAD COIL HOLDER

INSTALLATION

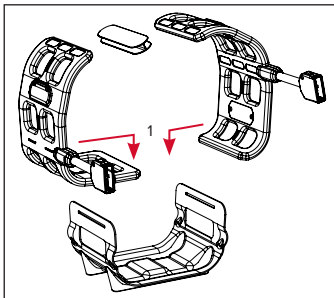
Fig. 2

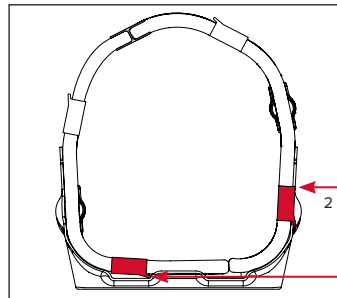
Fig. 3

1. Place 2 identical coils (per the Head Holder Compatibility Chart) into the MR Head Coil Holder.
 2. Ensure the Coil Amplifiers engage with the MR Head Coil Holder as shown.
- ! NOTE !** The coils will not be symmetric when installed in the head coil holder.

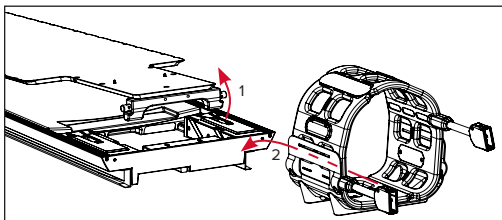
Fig. 4

1. Lift head end of the Insight Overlay.
2. Install Insight Head Coil Holder between the Insight Overlay and the MAGNETOM patient table.

RT-4700-03 INSIGHT MR BRIDGE WITH BODY COIL HOLDERS

SYSTEM COMPONENTS

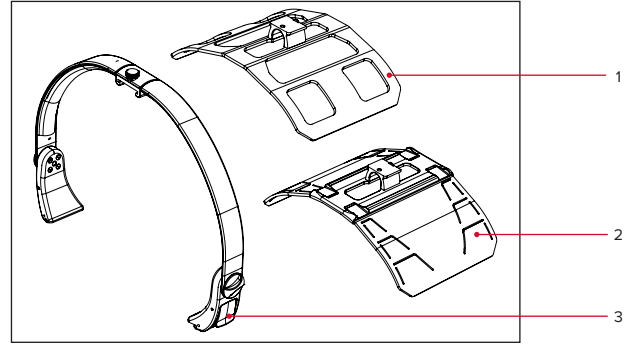
Fig. 5

1. Insight Body 6 Coil Holder
2. Insight Body 18 Coil Holder
3. Insight MR Bridge

BODY COIL HOLDER COMPATIBILITY CHART

MRI Scanner and Body Coil	Body 6 Coil Holder	Body 18 Coil Holder
MAGNETOM Aera 1.5T Body 6 Long PN: 10839370	✓	
MAGNETOM Aera 1.5T Body 18 Long PN: 10835255		✓
MAGNETOM Skyra and MAGNETOM Vida 3T Body 6 PN: 10654289	✓	
MAGNETOM Skyra and MAGNETOM Vida 3T Body 18 Long PN: 10835253		✓
MAGNETOM Altea 1.5T Body 6 Long PN: 10839370	✓	
MAGNETOM Altea 1.5T Body 18 Long PN: 10835255		✓
MAGNETOM Lumina 3T Body 6 Long PN: 10654289	✓	
MAGNETOM Lumina 3T Body 18 Long PN: 10835253		✓
Other Coils		✓ *

* May require additional Velcro® strapping not supplied with device.

INSIGHT BODY 6 COIL HOLDER

INSTALLATION

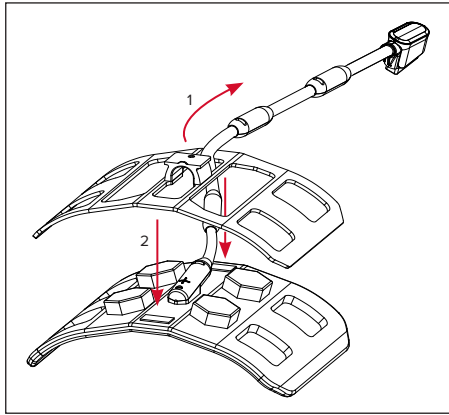
Fig. 6

1. Insert Body 6 cord through corresponding Coil Holder hole.
2. Attach center Velcro of the Body 6 coil to the Insight Coil Holder.

INSIGHT BODY 18 COIL HOLDER

INSTALLATION

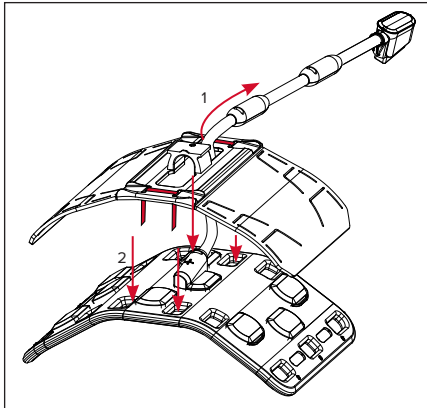
Fig. 7

1. Insert Body 18 cord through corresponding Body Coil Holder hole.
2. Pull the Velcro straps through holes in Coil.

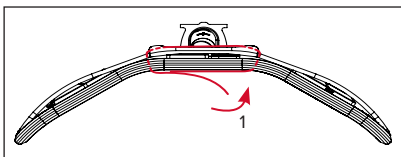
Fig. 8

1. Secure Coil with Velcro straps.

INSIGHT MR BRIDGE WITH BODY COIL HOLDERS

INSTALLATION

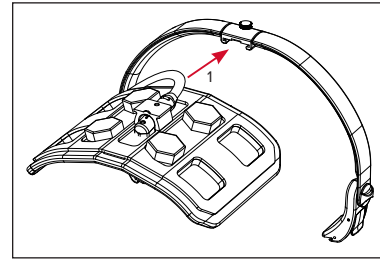
Fig. 9

1. Install the Body Coil Holder into the Bridge Clamp.

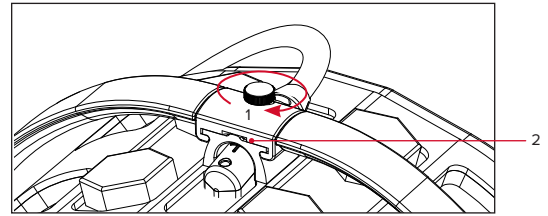
Fig. 10

1. Screw down the Bridge Clamp Knob.
2. Ensure the interlock tab is recessed into the Bridge Clamp.

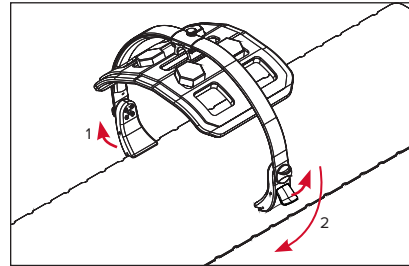
Fig. 11

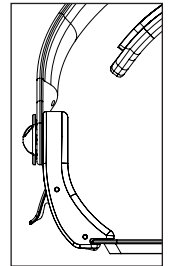
Fig. 12

1. Ensure both Clamp Handles are open.
2. Install the MR Bridge to the overlay by aligning one clamp body and then the other.

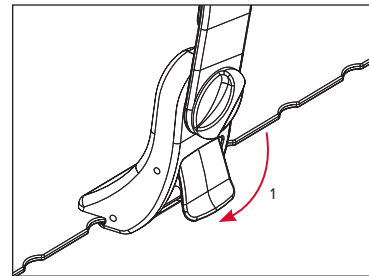
Fig. 13

1. Close both Clamp Handles to secure the MR Bridge to the overlay lip.

POSITIONING

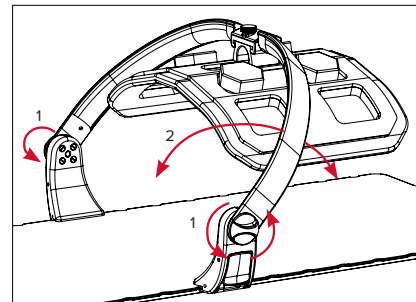
Fig. 14

1. Unscrew side knobs.

! NOTE ! Only 1 or 2 turns are required to disengage the pivot. Forcing the pivot past the stop may damage the mechanism.

2. Position height of MR Bridge.

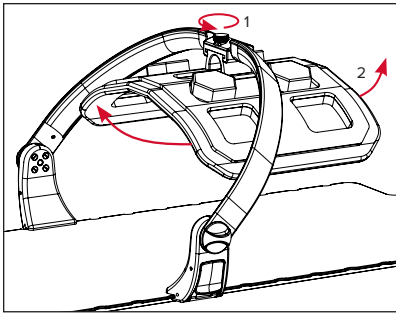
Fig. 15

1. Unscrew top knob.
2. Position angle of the Body Coil Holder.

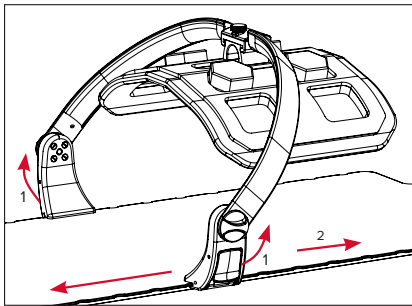
Fig. 16

1. Flip up side clamps to unlock.
 2. Slide bridge to desired location.
- Ensure the coil is as close to the patient's body as possible without touching.

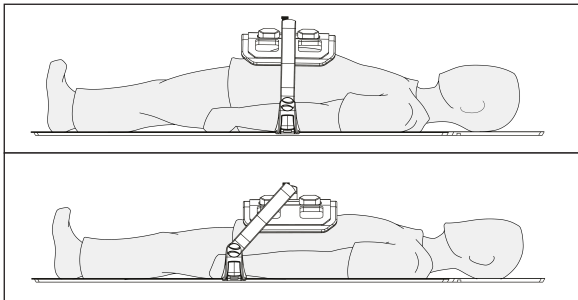
Fig. 17

To position closest to patient's body, may need to slide bridge back and pivot lower.

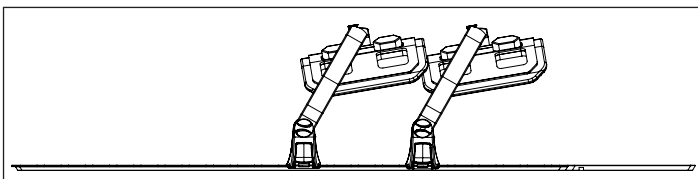
Fig. 18

Two (2) Insight MR Body Coil Holders can overlap for wider field of views.

MAINTENANCE

CLEANING THE SYSTEM

The device can be cleaned with a mild, non-abrasive cleaning or disinfecting solution. To clean, apply solution to clean cloth and wipe the surface. Visually inspect the device, if it is not clean, repeat the previous cleaning steps until visually clean. Use a clean cloth moistened with water to wipe the device to remove any cleaning agent residue. To dry, wipe the device with a clean, dry cloth. The following cleaning material has been tested and found to be appropriate for cleaning the device.

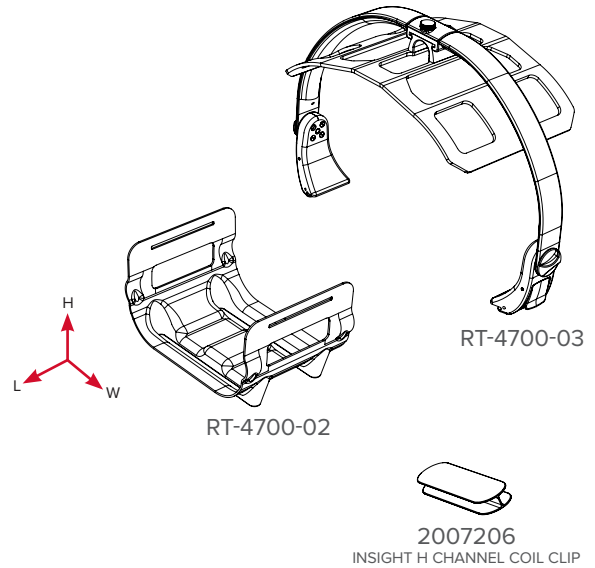
- Water
- Soap and Water

DISINFECTING THE SYSTEM

To disinfect the device surface, refer to specific instructions from the cleaning agent manufacturer.

- Isopropyl Alcohol
- 10% Clorox® Bleach Solution

SPECIFICATIONS

DIMENSIONS

RT-4700-02

Length (L): 24 cm [19.4 in]
 Width (W): 32.7 cm [12.9 in]
 Height (H): 19 cm [7.6 in]

RT-4700-03

Length (L): 33 cm [12.3 in]
 Width (W): 66 cm [26 in]
 Height (H): 46 cm [18 in]

Insight is a trademark of Qfix.

Siemens, MAGNETOM Aera, MAGNETOM Skyra, MAGNETOM Vida are registered trademarks of Siemens.

Velcro is a registered trademark of The Velcro Company.

Clorox is a registered trademark of The Clorox Company.

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