



positioning
patients for life.®

PRODUCT GUIDE & USER MANUAL

8002585

SofTouch™ Narrow Foot Block

For Philips®





| | | | | |
|----|-----|---|--|--|
| EC | REP | Advena Limited Tower Business Centre, 2nd Flr Tower Street, Swatar, BKR 4013 Malta | | Made in the USA by Qfix 440 Church Rd, Avondale, Pennsylvania, USA +1 610-268-0585 www.Qfix.com |
|----|-----|---|--|--|

SofTouch is a trademark of Qfix.

Philips® is a registered trademark of Philips.

Clorox® is a registered trademark of The Clorox Company.

TABLE OF CONTENTS

| | |
|--|----|
| GENERAL PRECAUTIONS..... | 4 |
| WARNING STATEMENTS | 4 |
| SERIOUS INCIDENTS..... | 4 |
| WARNING LABELS & DESCRIPTIONS | 4 |
| INTENDED USE..... | 5 |
| FEATURES | 6 |
| SYSTEM COMPONENTS..... | 6 |
| OPERATING INSTRUCTIONS..... | 7 |
| INSTALLATION..... | 7 |
| REMOVAL..... | 8 |
| MAINTENANCE..... | 9 |
| CLEANING INSTRUCTIONS | 9 |
| SPECIFICATIONS..... | 10 |
| DIMENSIONS..... | 10 |
| SETUP SHEET | 11 |
| 8002585 SOFTOUCH™ NARROW FOOT BLOCK FOR PHILIPS..... | 11 |

GENERAL PRECAUTIONS

WARNING STATEMENTS

! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT QFIX AT +1 484-720-6054 OR TECHSUPPORT@QFIX.COM.

! WARNING ! IT IS THE USER'S RESPONSIBILITY TO PERIODICALLY INSPECT THE UNIT FOR DAMAGE AND ATYPICAL WEAR.

! NOTE ! The SofTouch™ devices must be fully supported by the couch top surface at all times.

SERIOUS INCIDENTS

Please report any serious incidents (e.g. incidents which result in or have the potential to result in death or serious injury) to both Qfix and your country's Competent Authority.

WARNING LABELS & DESCRIPTIONS

Refer to Qfix.com for a listing of symbols and their definitions.

INTENDED USE

This device is intended to immobilize, position and reposition patients undergoing radiation therapy.

! NOTE ! United States Federal law restricts this device to sale by or on the order of a physician.

PATIENT TARGET GROUPS


Patients undergoing radiation therapy or diagnostic imaging procedures.

INTENDED USERS

The intended user for the products is a person qualified in accordance with the requirements of the regulatory region.

FEATURES

SYSTEM COMPONENTS

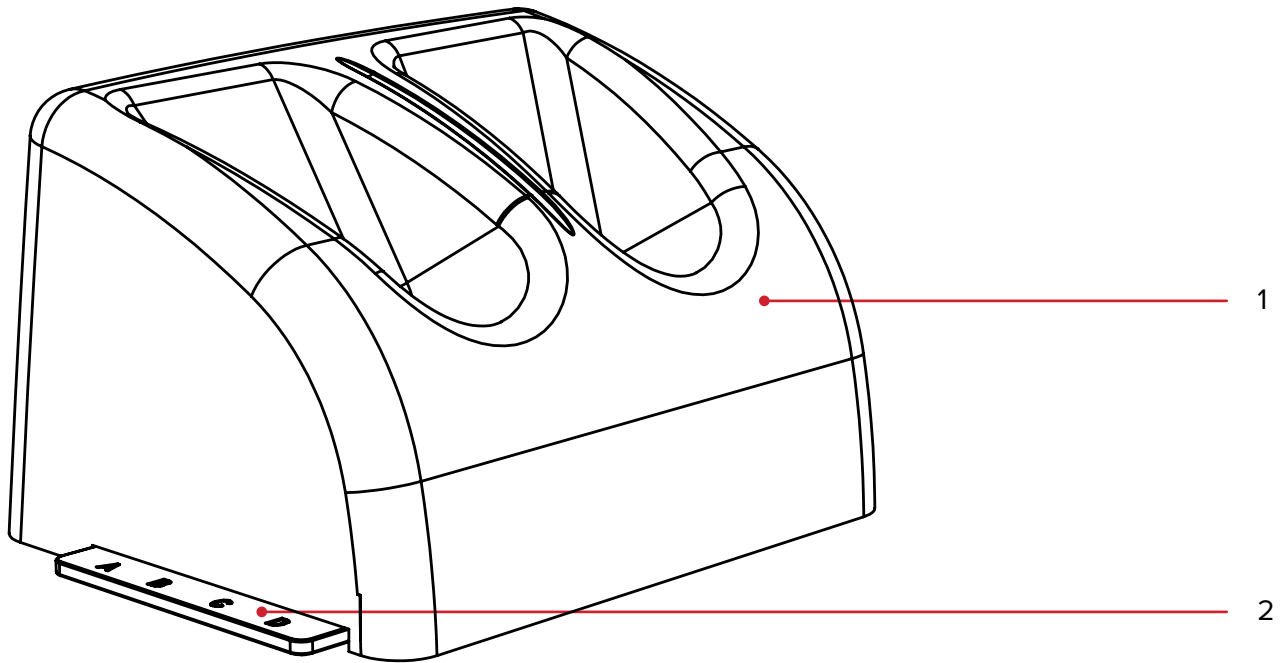

1. FOOT BLOCK

The SofTouch Foot Block is designed to provide precise, repeatable positioning and are contoured to relieve pressure point stress and improve patient comfort. It is lightweight and easy to clean.

2. INDEXING PLATES

The SofTouch devices contain Indexing Plates attached to them which locate to any device using a 2-pin or 3-pin locating bar.

OPERATING INSTRUCTIONS

INSTALLATION

Place the SofTouch™ device onto the couch top using the Indexing Plate to index to the locating bar.



OPERATING INSTRUCTIONS

REMOVAL

To remove the SofTouch devices, lift them up and away from the locating bar and off of the couch top.

MAINTENANCE

CLEANING INSTRUCTIONS

The device can be cleaned with a mild, non-abrasive cleaning or disinfecting solution. To clean, apply solution to clean cloth and wipe external surfaces.

The following cleaning materials have been tested and found to be appropriate for cleaning the SofTouch™ device:

- Water
- A 10% Clorox® Bleach Solution
- Isopropyl Alcohol
- Soap and Water

To disinfect the SofTouch device, refer to the specific instructions from the cleaning agent manufacturer.


SPECIFICATIONS

DIMENSIONS

Length (L): 28 cm [11 in]

Width (W): 36 cm [14.2 in]

Height (H): 20 cm [8 in]

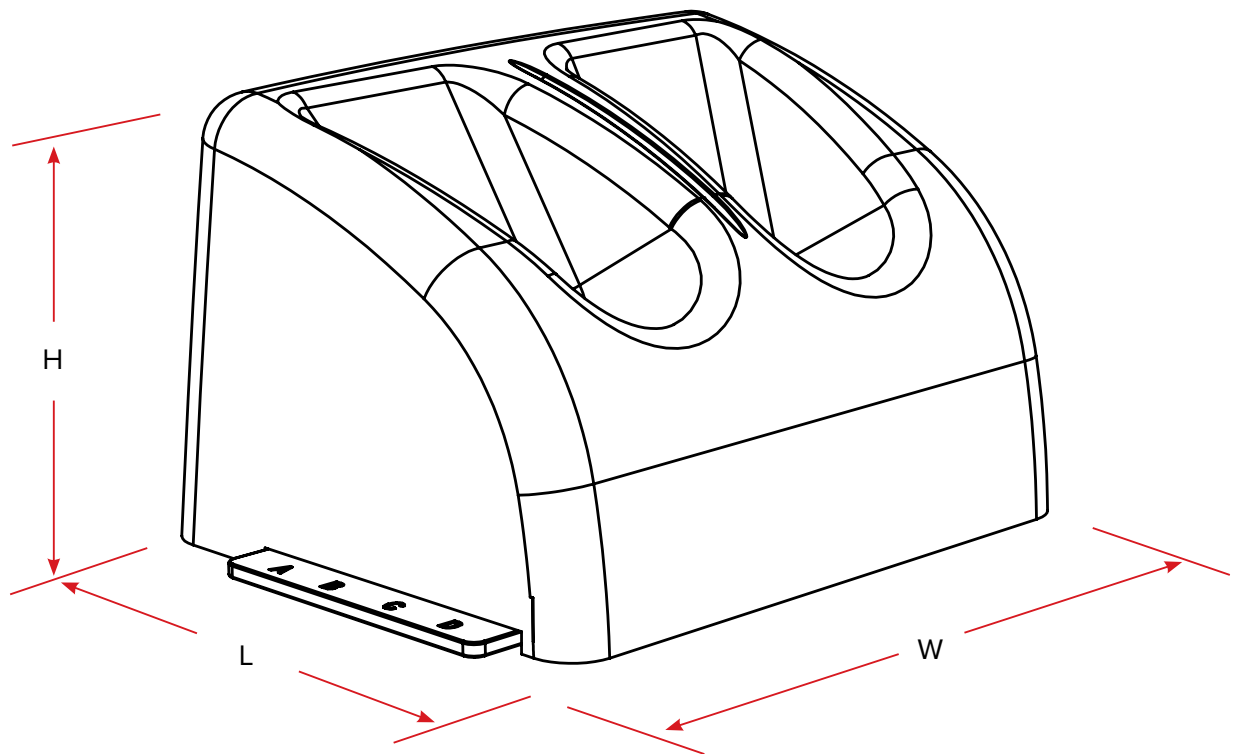

SETUP SHEET

8002585 SOFTOUCH™ NARROW FOOT BLOCK FOR PHILIPS

Patient Name:


Patient ID #:

Setup by:

Physician:

Date:

Comments:

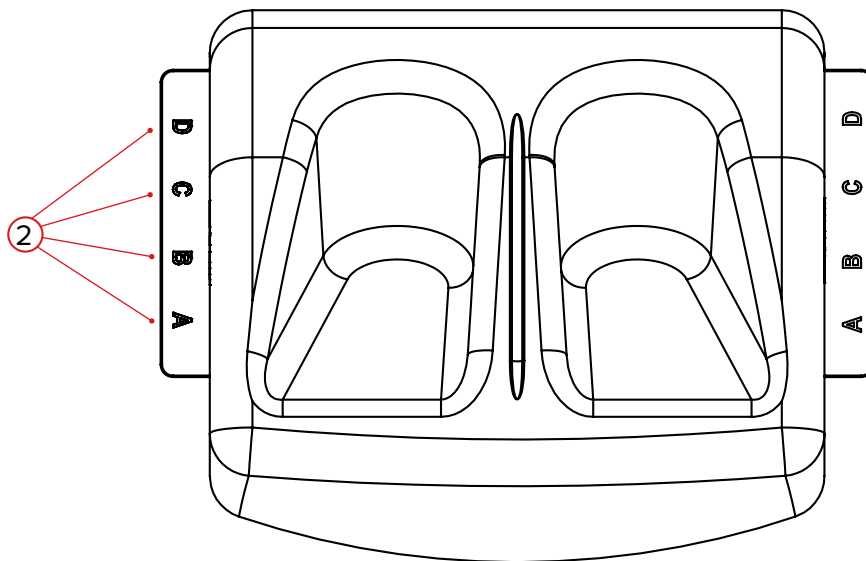

SOFTOUCH™ FOOT BLOCK LOCATION


1. Location of locating bar on couch top:


2. Location of SofTouch Foot Block on locating bar (circle one): **A** **B** **C** **D**

Notes:



440 Church Road
Avondale, PA 19311 USA
www.Qfix.com

 +1 610.268.0585 / 800.526.5247

 +1 610.268.0588 / 800.831.8174

 sales@Qfix.com