



ADDENDUM

RT-5100-01

Symphony[®] Standard Transfer Device


Instructions for Use when Cantilevering

OPERATING INSTRUCTIONS

This addendum applies specifically to the use of the Symphony® Standard Transfer Device in a cantilevered application. The content supplements the Symphony Standard Transfer Device Product Guide & User Manual 2007235. Prior to use, read and understand the Product Guide & User Manual in addition to this addendum.

ADDITIONAL WARNING LABELS & DESCRIPTIONS

Refer to Symphony Standard Transfer Device Product Guide & User Manual (PN 2007235) or Qfix.com for a listing of symbols and their definitions.

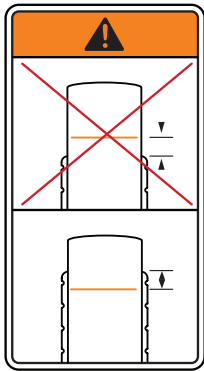

CANTILEVER WARNING

! WARNING ! DO NOT EXTEND THE SYMPHONY STANDARD TRANSFER DEVICE OFF THE END OF THE SUPPORTING DEVICE BEYOND THE ORANGE CANTILEVER LINE. DOING SO MAY CAUSE PATIENT OR PERSONAL INJURY.

Ensure that the device does not move when the patient is loaded or unloaded. Using an appropriate indexing adapter is highly recommended.



OR



DO NOT STEP LINE

Standing or sitting on the transfer device beyond the cantilever line may damage the device or cause injury.

ADDITIONAL PRECAUTIONS

! WARNING ! THE SYMPHONY STANDARD TRANSFER DEVICE SHALL NOT BE USED IN A CANTILEVERED POSITION UNLESS THE CANTILEVER LINE AND ASSOCIATED WARNING LABELS ARE PRESENT ON THE DEVICE.

! WARNING ! WHEN CANTILEVERING THE DEVICE, THE PATIENT MUST BE ORIENTED WITH HIS OR HER HEAD TOWARD OR ON THE CANTILEVERED END. THE PATIENT SHALL NOT EXTEND BEYOND THE END OF THE DEVICE.

! WARNING ! ENSURE THAT THE UNDERLYING SUPPORTING DEVICE OR RECEIVING SURFACE IS CAPABLE OF SUPPORTING THE CANTILEVERED LOAD OF THE TRANSFER DEVICE, PATIENT, AND ANY ADDITIONAL EQUIPMENT.

ADDITIONAL INSTRUCTIONS

1. Place the transfer device on the supporting device in the cantilevered position, taking care not to cantilever beyond the indicated line.

! WARNING ! THE TRANSFER DEVICE MUST BE IN POSITION PRIOR TO LOADING THE PATIENT. DO NOT ATTEMPT TO PERFORM A PATIENT TRANSFER INTO OR OUT OF THE CANTILEVERED POSITION USING THE TRANSFER DEVICE.

2. Assist the patient onto the transfer device by guiding the patient to sit near the middle of the device. Rotate the patient into a supine position with their head toward or on the cantilevered end. Refer to WARNING LABELS & DESCRIPTIONS and ADDITIONAL PRECAUTIONS.
3. To unload the patient, assist the patient to sit up onto the non-cantilevered portion of the device and then to step to the floor.

2007908_EN_B