



positioning
patients for life.®

MAINTENANCE GUIDE

2006857 – O-ring, 1.5" – Pack of 10
Arm Support Replacement Band



MAINTENANCE

ARM SUPPORT REPLACEMENT BAND

! NOTE ! The O-ring should be replaced every 6 months.

1. Remove the Arm Support from Device.
2. Flip the Arm Support over to expose the O-ring. (Fig. 1)
3. Pull up from the side of the O-ring, until it is removed from the Arm Support. (Fig. 2–5)
4. Install a new O-ring and reattach the Arm Support.



FIG. 1



FIG. 2



FIG. 3

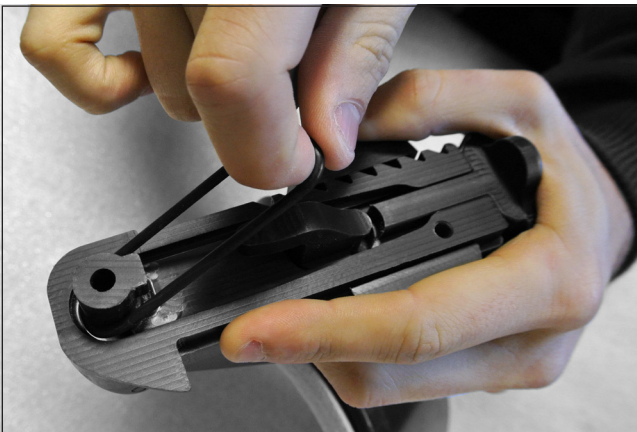


FIG. 4

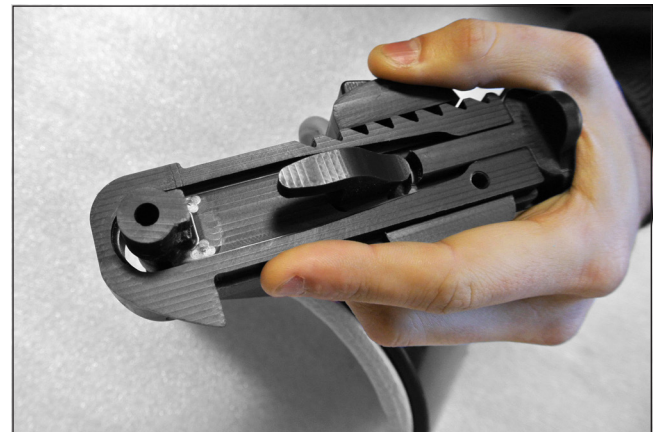


FIG. 5