



INSTALLATION MANUAL

RT-4546H-02 Head Cushion Adapter Plate
for the Access™ Supine

RT-4552P Prone Head Cushion

RT-4552A Head Cushion Adapter
for Silverman Pins

GENERAL PRECAUTIONS

WARNING STATEMENTS

! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT QFIX AT +1 484-720-6054 OR TECHSUPPORT@QFIX.COM.

MRI SAFETY INFORMATION

 Non-clinical testing has demonstrated the **Head Cushion Adapter Plate for the Access™ Supine, Prone Head Cushion** and the **Head Cushion Adapter for Silverman Pins** are MR Safe.

WARNING LABELS & DESCRIPTIONS

Refer to Qfix.com for a listing of symbols and their definitions.



MR SAFE

An item that poses no known hazards resulting from exposure to any MR environment. MR Safe items are composed of materials that are electrically non-conductive, nonmetallic and nonmagnetic.

RT-4546H-02

HEAD CUSHION ADAPTER PLATE FOR THE ACCESS™ SUPINE

Position the Head Cushion Adapter to the Access™ Supine as shown in Figure 1 and Figure 2 by inserting the plate pins on the Head Pad Adapter Plate into the appropriate holes on the Access™ Supine device.

! NOTE ! Refer to the Product Guide and User Manual for the Access™ Supine (P/N 2004394) for operating instructions, patient setup sheet, product features, specifications, warnings, cautions, and other general precautions related to the use of the Access™ Supine.

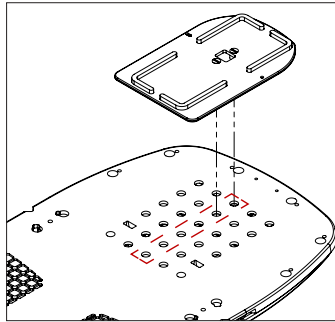


Figure 1

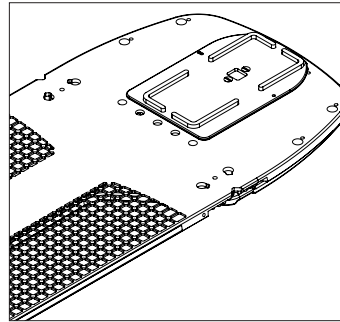


Figure 2

RT-4552A AND RT-4552P

HEAD CUSHION ADAPTER FOR SILVERMAN PINS AND PRONE HEAD CUSHION

Position the Head Cushion Adapter Plate to the device as shown in Figure 3 by pressing the holes on the Head Cushion Adapter Plate over the Silverman Pins on the device.

Place the Prone Head Cushion onto the Head Cushion Adapter (Fig. 4).

! NOTE ! Refer to the Product Guide and User Manual for the applicable device for operating instructions, product features, specifications, warnings, cautions, and other general precautions related to the use of the device.

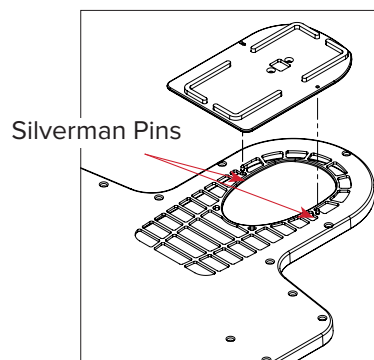


Figure 3

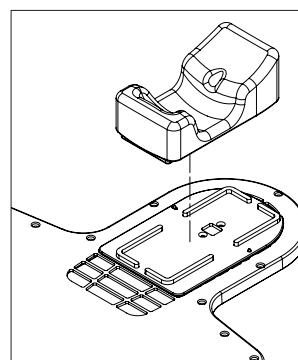


Figure 4



Made in the USA by

Qfix

440 Church Rd, Avondale, Pennsylvania, USA

+1 610-268-0585 www.Qfix.com