



# SERVICE INSTRUCTIONS

## SVC-5100-113

### AirDrive™ Cable Tie Mount Kit

#### PURPOSE

Replace cable ties and cable tie mounts for wire routing in AirDrive™ electrical systems. Four-way mounts allow ties to be inserted from any side.

#### WARNING STATEMENTS

- ⚠ ! WARNING ! RECOMMENDED MAINTENANCE AND SERVICE, AS WELL AS USE OF ONLY QFIX-PROVIDED ACCESSORIES, COMPONENTS, AND REPLACEMENT PARTS, ARE NECESSARY TO ENSURE THE SAFETY, PERFORMANCE, AND MRI COMPATIBILITY OF THE PRODUCT(S), AS WELL AS TO MAINTAIN APPLICABLE WARRANTIES.**
- ⚠ ! WARNING ! ALL MAINTENANCE AND SERVICE PERFORMED ON THE AIRDRIVE SYSTEM, AIRSHUTTLE™ DEVICES, AND ALL ACCESSORIES SHOULD BE CONDUCTED OUTSIDE MRI ENVIRONMENTS.**

#### DIRECTIONS

- If necessary, remove cable tie mount with a flat blade, taking care not to damage any cables, wires, or other components.
- Clean area with isopropyl alcohol or other cleaning solution. Allow to fully dry.
- Place new cable tie mount and add a cable tie to secure a wire bundle.

#### ESTIMATED TIME TO COMPLETE

5–10 minutes per cable tie and cable tie mount unit.

## KIT CONTENTS

Part Number	Quantity	Part Name
3001350	12	Cable Tie, 0.086" Width, 8" Long
3002026	6	Cable Tie Mount, 4 Way, 0.20" Tie Width, ¾" Long

## REQUIRED TOOLS

- Isopropyl Alcohol (or equivalent cleaning solvent)
- Gloves and Rags
- Flat blade or scraping tool



440 Church Road  
Avondale, PA 19311 USA  
[www.Qfix.com](http://www.Qfix.com)

+1 484.720.6053 / 800.526.5247

+1 610.268.0588 / 800.831.8174

[sales@Qfix.com](mailto:sales@Qfix.com)

AirDrive and AirShuttle are trademarks of Qfix.

2008589\_EN\_B  
2022-01