



SERVICE INSTRUCTIONS


SVC-5100-112


AirDrive Trolley™ Fuse Replacement

PURPOSE

Replace fuse components in the AirDrive Trolley.


WARNING STATEMENTS

-  **! WARNING ! RECOMMENDED MAINTENANCE AND SERVICE, AS WELL AS USE OF ONLY QFIX-PROVIDED ACCESSORIES, COMPONENTS, AND REPLACEMENT PARTS, ARE NECESSARY TO ENSURE THE SAFETY, PERFORMANCE, AND MRI COMPATIBILITY OF THE PRODUCT(S), AS WELL AS TO MAINTAIN APPLICABLE WARRANTIES.**

-  **! WARNING ! ALL MAINTENANCE AND SERVICE PERFORMED ON THE AIRDRIVE™ SYSTEM, AIRSHUTTLE™ DEVICES, AND ALL ACCESSORIES SHOULD BE CONDUCTED OUTSIDE MRI ENVIRONMENTS.**

DIRECTIONS

- Lift the trolley base cover using the T27 tamper resistant screwdriver and the bungee cords to help hold up the base cover.
- Use the diagram to determine location of included fuses.

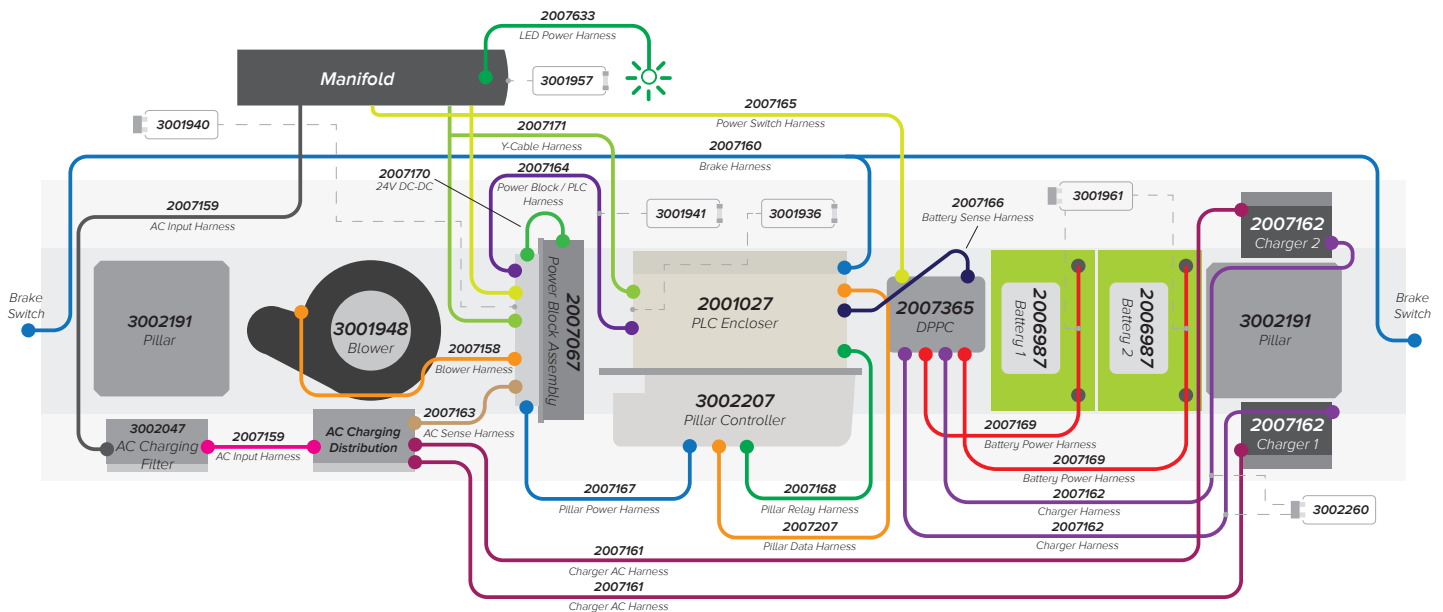

ESTIMATED TIME TO COMPLETE

30–60 minutes hands on to replace fuses in AirDrive Trolley.

KIT CONTENTS

Part Number	Quantity	Part Name
3001936	1	Fuse, 2A Glass, PLC, AirDrive Trolley
3001940	3	Fuse, ATO, 25A, Power Block, AirDrive Trolley
3001941	1	Fuse, Fast, 250mA, Power Block, AirDrive Trolley
3001957	2	Fuse, AC, 250V, 10A, AirDrive Trolley
3001961	2	Fuse, ATO, 30A, Battery, AirDrive Trolley
3002260	2	Fuse Automotive Fast Acting 7.5A

REQUIRED TOOLS

- Screwdriver with T27 tamper resistant bit
- Bungee cords (x2) to hold up the trolley base cover



440 Church Road
Avondale, PA 19311 USA
www.Qfix.com

+1 484.720.6053 / 800.526.5247
+1 610.268.0588 / 800.831.8174
sales@Qfix.com

AirDrive, AirDrive Trolley, and AirShuttle are trademarks of Qfix.

2008590_EN_B
2022-01