




positioning
patients for life.®

SERVICE INSTRUCTIONS

RT-4546AW-01 (8003049)

Access™ Wrist Support Cam
Component

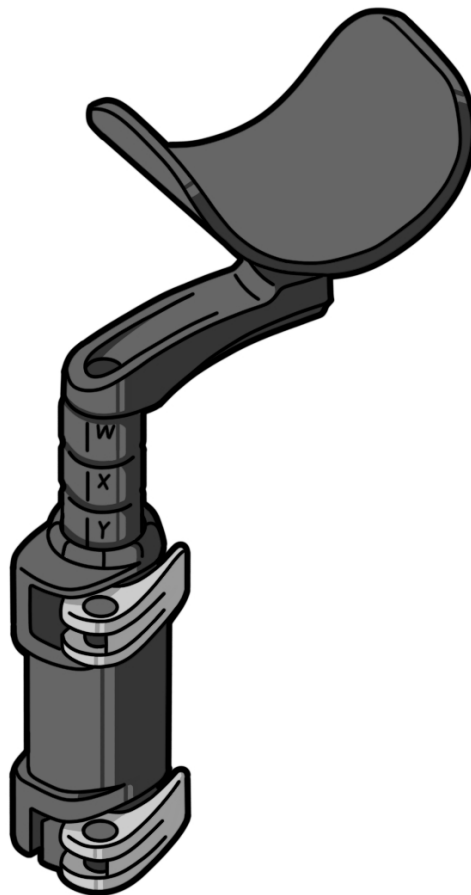

PURPOSE

To replace the bottom cam component in the Access™ Wrist Support.

WARNING STATEMENTS

! WARNING ! NO MODIFICATION OF THIS EQUIPMENT IS ALLOWED. IF ANY PART OF THIS DEVICE EXPERIENCES A CATASTROPHIC LOAD, APPEARS DAMAGED OR FUNCTIONS IMPROPERLY, DISCONTINUE USE IMMEDIATELY AND CONTACT QFIX AT +1 484-720-6054 OR TECHSUPPORT@QFIX.COM.

! WARNING ! NEVER LIFT OR CARRY THE DEVICE BY HOLDING THE WRIST SUPPORTS.

TRAPPING ZONES

- Trapping zones may occur when adjusting the Access™ Wrist Support height.
- Trapping zones may occur when adjusting the Access™ Wrist Support angle.

MRI SAFETY INFORMATION



Non-Clinical testing has demonstrated the Access™ Wrist Supports are MR Conditional. These devices may be used in an MR system meeting the following conditions:

- The Access Wrist Supports may be placed in the bore of an MR system having a static Magnetic field of 1.5T or less.
- The Access Wrist Supports must be used with the MR Safe version of the Access™ Supine (RT-4546MRI) in accordance with its intended use prior to scanning.
- The MR Safe version of the Access Supine (RT-4546MRI) is clearly labeled with the following symbol:



DIRECTIONS

1. To replace the bottom section of the Access Wrist Support, open the upper lever. (Fig. 1)
2. Pull firmly on the post of the Access Wrist Support to separate the cup from the bottom section. (Fig. 2)
3. Slide the post of the Access Wrist Support into the new bottom section. (Fig. 3)
4. Close the upper lever to lock in place. (Fig. 4)



FIG. 1



FIG. 2



FIG. 3



FIG. 4

ESTIMATED TIME TO COMPLETE

Less than two minutes hands on to replace the bottom cam component of the Access™ Wrist Support.

KIT CONTENTS

Part Number	Quantity	Part Name
2008591	1	Repair kit, Cam component, Access Wrist Support



440 Church Road
Avondale, PA 19311 USA
www.Qfix.com

+1 610.268.0585 / 800.526.5247

+1 610.268.0588 / 800.831.8174

sales@Qfix.com

Access is a trademark of Qfix.

2008610_EN_A
2020-09