



# SERVICE INSTRUCTIONS

## 8003078

## FlexLock™ Knee Support Pin Replacement Kit

### PURPOSE

Replace damaged pins on the shell of the FlexLock Knee Support.

### WARNING STATEMENTS

**! WARNING ! RECOMMENDED MAINTENANCE AND SERVICE, AS WELL AS USE OF ONLY FLEXLOCK ACCESSORIES, COMPONENTS, AND REPLACEMENT PARTS, ARE NECESSARY TO ENSURE THE SAFETY, PERFORMANCE, AND MRI COMPATIBILITY OF THE PRODUCT(S), AS WELL AS TO MAINTAIN APPLICABLE WARRANTIES.**

**! NOTE !** Refer to the Product Guide and User Manual for the FlexLock Knee Support (P/N 2008425) for operating instructions, product features, specifications, warnings, cautions, and other general precautions related to the use of the FlexLock Knee Support Device.

### ESTIMATED TIME TO COMPLETE

10–20 minutes hands-on

### KIT CONTENTS

Part Number	Quantity	Part Name
2008393	2	Knee Shell Pin, Small
2008328	2	Knee Shell Pin, Large
3002219	4	PHS M8 x 12 mm, Black Nylon, Glass Filled

### REQUIRED TOOLS

- Flathead (Wide Blade) Screwdriver
- Pliers or Adjustable Wrench

# DIRECTIONS

1. Remove shell from base.
2. Using a flathead screwdriver and pliers/adjustable wrench, remove all pins and screws. (Fig. 1)



Fig. 1

3. Using a flathead screwdriver and pliers/adjustable wrench, install small pins (2008393) on end with chamfered corners. (Fig. 2)

**! NOTE ! DO NOT OVERTIGHTEN SCREWS.**



Fig. 2

4. Using a flathead screwdriver and pliers/adjustable wrench, install large pins (2008328) on end with rounded corners. (Fig. 3)

**! NOTE ! DO NOT OVERTIGHTEN SCREWS.**

5. Install shell onto base and ensure it sits flush.



Fig. 3



440 Church Road  
Avondale, PA 19311 USA  
[www.Qfix.com](http://www.Qfix.com)

+1 610.268.0585 / 800.526.5247

+1 610.268.0588 / 800.831.8174

[sales@Qfix.com](mailto:sales@Qfix.com)

FlexLock is a trademark of Qfix.

2008743\_EN\_A  
2021-02