PRODUCT GUIDE & USER MANUAL

RT-4700-04 Insight™ MRI Overlay,
Siemens® Free.Max™

 EC REP Advena Limited Tower Business Centre, 2nd Flr Tower Street, Swatar, BKR 4013 Malta	UK Responsible Person QServe Group UK, Ltd. 49 Greek Street Soho, London W1D 4EG United Kingdom
 Made in the USA by Qfix 440 Church Rd, Avondale, Pennsylvania, USA +1 484-720-6053 www.Qfix.com	CH REP Raditec Medical AG Schlossberg 5a 5454 Bellikon Switzerland

2009105_ML_D / 2024-04

EN	English	NL	Dutch (Nederlands)
DE	German (Deutsch)	PT-EU	Portuguese (Português)
ES	Spanish (Español)	RO	Romanian (Română)
FR	French (Français)	SV	Swedish (Svenska)
IT	Italian (Italiano)		
JA	Japanese (日本語)		

EN: English

GENERAL PRECAUTIONS

WARNING STATEMENTS

! WARNING ! No modification of this equipment is allowed. If any part of this device experiences a catastrophic load, appears damaged or functions improperly, discontinue use immediately and contact Qfix at +1 484-720-6053 or techsupport@qfix.com.

! NOTE ! Headphones are **not compatible when using thermoplastic masks**. The use of earplugs are still permitted.

ADDITIONAL WARNINGS AND PRECAUTIONS

! WARNING ! DO NOT carry a patient on the MRI Overlay.

! WARNING ! The use of additional devices may create additional trapping zones not described in this IFU.

Inspect product for damage before each use.

SERIOUS INCIDENTS

Please report any serious incidents (e.g. incidents which result in or have the potential to result in death or serious injury) to both Qfix and your country's Competent Authority.

MRI SAFETY INFORMATION

 Non-clinical testing has demonstrated the **Insight MR Overlay** is MR Safe. This device may be used in an MR system meeting the following conditions:

- Static magnetic field of 3T or less.

! WARNING ! Use of unapproved MR accessories may result in:

- Injury to patient
- Patient burns

Use only proven MR Safe or MR Conditional accessories, tested and approved for your MR system.

Consider and evaluate possible distortions of the MR images used, particularly when fusing with CT datasets.

Consider the MR compatibility of accessories prior to use on your MR system.

WARNING LABELS & DESCRIPTIONS

Refer to Qfix.com for a listing of symbols and their definitions.

	MR SAFE An item that poses no known hazards resulting from exposure to any MR environment. MR Safe items are composed of materials that are electrically non-conductive, nonmetallic and nonmagnetic.
	LOAD RATING DO NOT exceed 243 kg (536 lb) uniformly distributed load when installed on a MR Table. When on a couch base the maximum safe working load is that of the couch base.

INTENDED USE

The Insight™ MR Overlay is an MR Safe patient support device. It is intended to allow patients to undergo diagnostic imaging in the same position as that of the treatment position.

! NOTE ! United States Federal law restricts this device to sale by or on the order of a physician.


PATIENT TARGET GROUPS

Patients undergoing radiation therapy or diagnostic imaging procedures.

INTENDED USERS

The intended user for the products is a person qualified in accordance with the requirements of the regulatory region.

FEATURES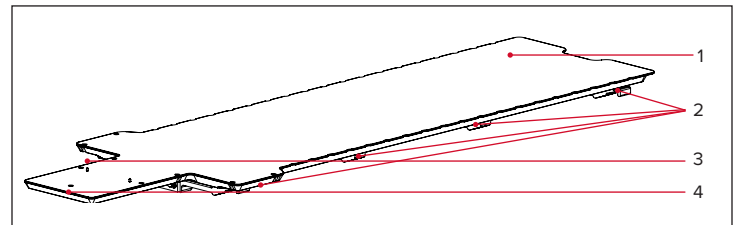

SYSTEM COMPONENTS

1. INSIGHT™ MR OVERLAY

The Insight MR Overlay mounts to compatible MR table tops and allows for use of patient positioning devices during MR scanning for radiotherapy.

2. MOUNTING HARDWARE

The Mounting Hardware comes pre-installed onto the Insight MR Overlay. The mounts secure the Insight MR Overlay to compatible MR cradles.

3. SILVERMAN PINS

The Silverman pins locate Silverman headrests to the Insight MR Overlay. These may be removed when not in use.

4. MASK ATTACHMENT HOLES

The mask attachment holes on the Insight MR Overlay are compatible with head-only or head and shoulders S-type thermoplastic masks.

MR COMPATIBILITY CHART





MRI Scanners	Insight Overlay RT-4700-04
MAGNETOM Free.Max	✓
Other MRs	*

* Not compatible with other MRI Scanners

OPERATING INSTRUCTIONS

OVERLAY INSTALLATION

1. Place the overlay onto the cradle by aligning the notch on the shoulder mounting hardware with the notch on the cradle.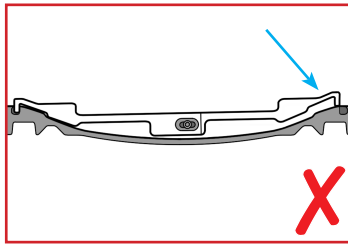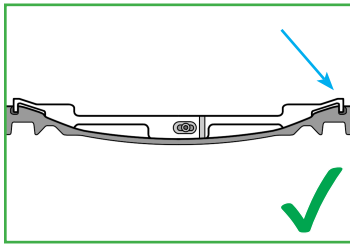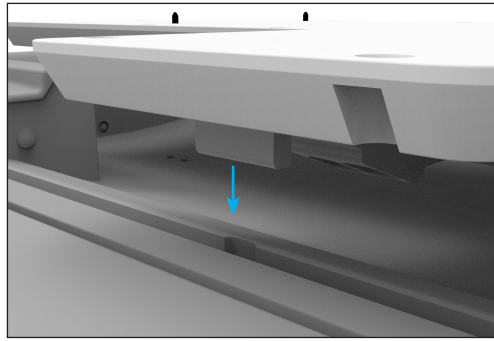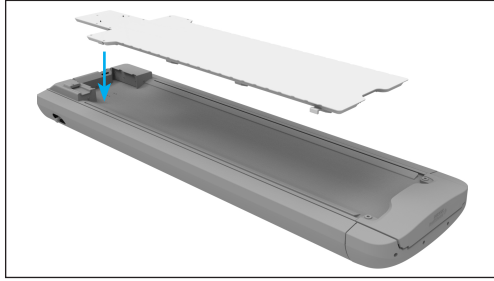

2. If Overlay does not fit into cradle or there is unintended movement, mounting hardware adjustments are required. Remove Overlay by lifting it off the cradle and see Installation Manual 2009122 for further instructions.

SILVERMAN PIN REMOVAL

1. Remove pins from Insight MR Overlay by unscrewing counterclockwise from the bottom of the device.
2. Once threads are fully disengaged, remove pins from locating holes.

MAINTENANCE

CLEANING THE SYSTEM

The device can be cleaned with a mild, non-abrasive cleaning or disinfecting solution. To clean, apply solution to clean cloth and wipe the surface. Visually inspect the device; if it is not clean, repeat the previous cleaning steps until visually clean. Use a clean cloth moistened with water to wipe the device to remove any cleaning agent residue. To dry, wipe the device with a clean, dry cloth. The following cleaning material has been tested and found to be appropriate for cleaning the device.


- Water
- Soap and Water

DISINFECTING THE SYSTEM

To disinfect the device surface, refer to specific instructions from the cleaning agent manufacturer.

- 10% Clorox® Bleach Solution
- Isopropyl Alcohol

SPECIFICATIONS

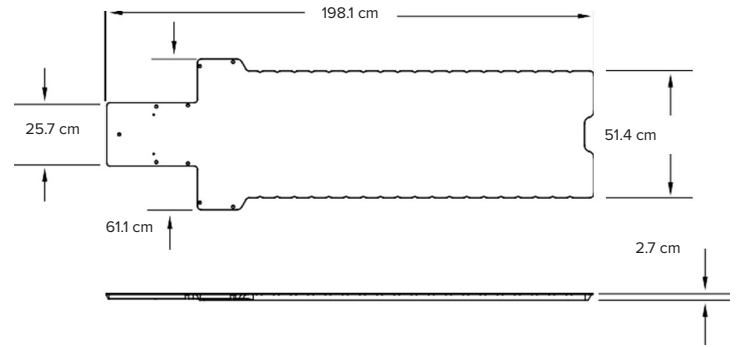DIMENSIONS

LENGTH (L): 198.1 cm [78 in]

WIDTH (W): 61.1 cm [24.06 in]

HEIGHT (H): 2.7 cm [1.07 in]

PARTS LIST

1. 2001443 – Silverman pin
2. 3001064 – Mounting Hardware Attachment Screw (SHCS M8 x 1.25 x 20 mm)

Insight is a trademark of Qfix.

Siemens and MAGNETOM Free.Max are registered trademarks of Siemens.

Clorox is a registered trademark of The Clorox Company.

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