# PRODUCT GUIDE & USER MANUAL

RT-4700-05 Insight™ MR Coil Holder,  
Siemens® Free.Max™

|                                                                                                                                                                                                          |                                                                                                                               |
|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------|
|  <p><b>EC REP</b><br/>Advena Limited<br/>Tower Business Centre, 2nd Flr<br/>Tower Street, Swatar, BKR 4013<br/>Malta</p> | <p><b>UK Responsible Person</b><br/>QServe Group UK, Ltd.<br/>49 Greek Street<br/>Soho, London W1D 4EG<br/>United Kingdom</p> |
| <p>Made in the USA by<br/>Qfix<br/>440 Church Rd,<br/>Avondale, Pennsylvania, USA<br/>+1 484-720-6053 www.Qfix.com</p>                                                                                   | <p><b>CH REP</b><br/>Raditec Medical AG<br/>Schlossberg 5a<br/>5454 Bellikon<br/>Switzerland</p>                              |

2009123\_ML\_E / 2024-04

|    |                    |       |                        |
|----|--------------------|-------|------------------------|
| EN | English            | NL    | Dutch (Nederlands)     |
| DE | German (Deutsch)   | PT-EU | Portuguese (Português) |
| ES | Spanish (Español)  | RO    | Romanian (Română)      |
| FR | French (Français)  | SV    | Swedish (Svenska)      |
| IT | Italian (Italiano) |       |                        |
| JA | Japanese (日本語)     |       |                        |

EN: English

## GENERAL PRECAUTIONS

### WARNING STATEMENTS

**! WARNING !** No modification of this equipment is allowed. If any part of this device experiences a catastrophic load, appears damaged or functions improperly, discontinue use immediately and contact Qfix at +1 484-720-6053 or techsupport@qfix.com.

### SERIOUS INCIDENTS

Please report any serious incidents (e.g. incidents which result in or have the potential to result in death or serious injury) to both Qfix and your country's Competent Authority.

### SAFETY INFORMATION

In order to operate the Insight Coil Holders accurately and safely, the operating personnel must have the necessary expertise in a hospital setting.

To ensure safe use of the Insight Coil Holders, it is recommended that the user is educated and trained on the safe operation of the system prior to use.

### MRI SAFETY INFORMATION

Consider and evaluate possible distortions of the MR images acquired. Establish a quality process in order to assess remaining distortions independent of system type, size of Field of View, and MR measurement parameters.

 Non-clinical testing has demonstrated the **Insight Coil Holder** is MR Safe. This device may be used in an MR system meeting the following conditions.

- Static magnetic field of .55t or less.
- With devices listed in the head coil holder compatibility chart

**! WARNING !** Use of unapproved MR accessories may result in:

- Injury to patient
- Patient burns

Use only proven MR Safe or MR Conditional accessories, tested and approved for your MR system.

Consider the MR compatibility of accessories prior to use on your MR system.

## ADDITIONAL PRECAUTIONS

Inspect product for damage before each use.

## WARNING LABELS & DESCRIPTIONS

Refer to [Qfix.com](http://Qfix.com) for a listing of symbols and their definitions.

### MR SAFE

An item that poses no known hazards resulting from exposure to any MR environment. MR Safe items are composed of materials that are electrically non-conductive, nonmetallic and nonmagnetic.

## INTENDED USE

The Insight Coil Holder is an MR Safe device intended to position a head coil to allow the patient to undergo diagnostic imaging in the same position as that of the treatment position.

**! NOTE !** United States Federal law restricts this device to sale by or on the order of a physician.


### PATIENT TARGET GROUPS

Patients undergoing radiation therapy or diagnostic imaging procedures.

### INTENDED USERS

The intended user for the products is a person qualified in accordance with the requirements of the regulatory region.

## SYSTEM COMPONENTS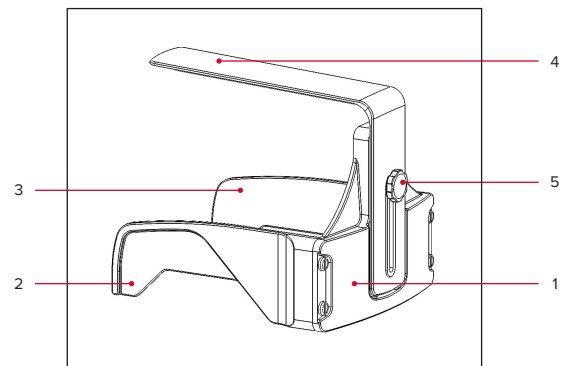

1. Back Plate
2. Right Side Wall
3. Left Side Wall
4. Coil Support
5. Adjustment Knob

## HEAD COIL HOLDER COMPATIBILITY CHART

| Coil                           | Head Coil Holder |
|--------------------------------|------------------|
| MAGNETOM Free.Max<br>Contour M | ✓                |
| Other Coils                    | *                |

\* Qfix does not recommend the use of other coils with the Head Coil Holder.


## OPERATING INSTRUCTIONS

### ASSEMBLY


1. Align Side Walls with Back Plate and fasten together with provided screws using Phillips-Head screwdriver.

**! NOTE !** Do not overtighten screws.

**! NOTE !** The side walls will only install in the correct orientation on the back plate.

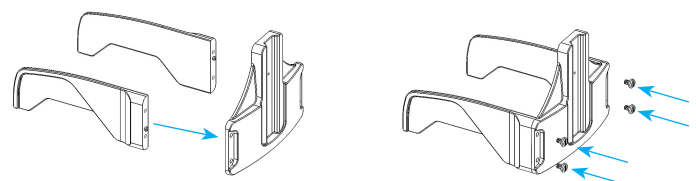
- Align Coil Support with Back Plate and then attach with Adjustment Knob.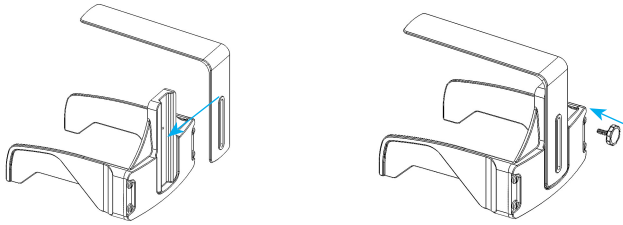


## INSTALLATION


- Place the MR Head Coil Holder onto the device. Ensure it sits flat on top of mask frame.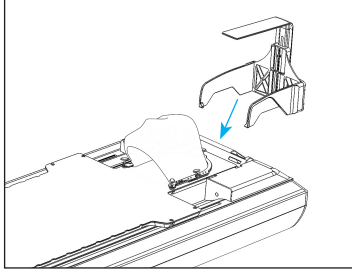

- Place coil (per the Head Holder Compatibility Chart) onto the MR Head Coil Holder.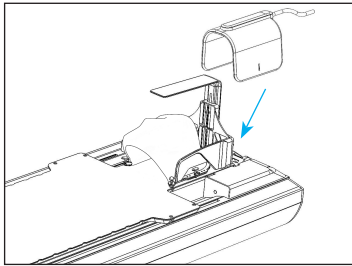

- To adjust the height, first loosen the Adjustment Knob then slide Coil Support to desired height. Tighten Adjustment Knob to lock the height.

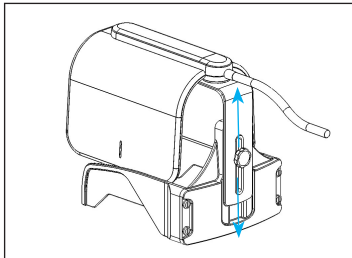- A Contour S coil or a Contour M coil may be placed under the head of the device.

## MAINTENANCE

### CLEANING THE SYSTEM

The device can be cleaned with a mild, non-abrasive cleaning or disinfecting solution. To clean, apply solution to clean cloth and wipe the surface. Visually inspect the device, if it is not clean, repeat the previous cleaning steps until visually clean. Use a clean cloth moistened with water to wipe the device to remove any cleaning agent residue. To dry, wipe the device with a clean, dry cloth. The following cleaning material has been tested and found to be appropriate for cleaning the device.




- Water
- Soap and Water

### DISINFECTING

The following products are recommended. Refer to specific instructions from the chemical agent manufacturer:

- Isopropyl Alcohol
- 10% Clorox® Bleach Solution

## SPECIFICATIONS

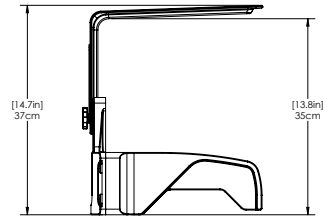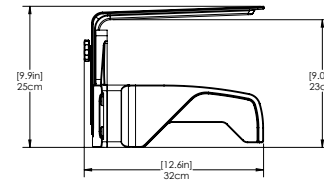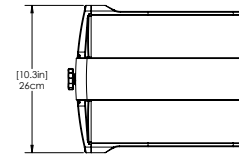## DIMENSIONS

Length (L): 32 cm [12.6 in]

Width (W): 26.3 cm [10.3 in]

Height (H): 25.2 cm [9.9 in] MIN – 37.4cm [14.7 in] MAX

## PARTS LIST

- 3002424 – Assembly Screw (PHS M8 x 1.25 x 16mm [0.63in])
- 2009124 – Adjustment Knob
- 2009117 – Left Side Wall
- 2009116 – Right Side Wall

Insight is a trademark of Qfix.

Siemens and MAGNETOM Free.Max are registered trademarks of Siemens.

Clorox is a registered trademark of The Clorox Company.

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