



positioning
patients for life.®

GUIDE FOR BEST RESULTS

2005445

Encompass RT–B889KYCF2

DIRECTIONS

2009209_EN_A
2023-11

INSTALLING ANTERIOR MASK PINS

1. Slide the anterior thermoplastic mask over the patient.
2. Press down to install the two most inferior pins first (Fig. 1).
3. Press down to install the two center pins (Fig. 2).
4. Press down to install the two most superior pins (Fig. 3).

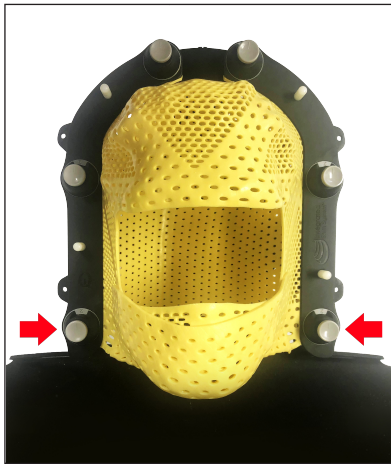


Fig. 1

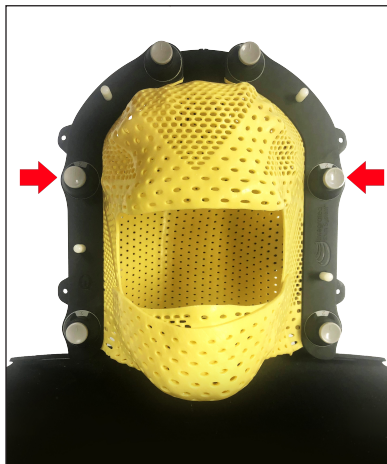


Fig. 2



Fig. 3

ACHIEVING PIN ALIGNMENT

1. To ease tension and encourage pin alignment, apply outward pressure onto the posterior mask closest to the inferior pins. (Fig. 4)

! NOTE ! In order to achieve alignment, this step may be required to be performed in conjunction with Step 2 from 'Installing Anterior Mask Pins'.



Fig. 4